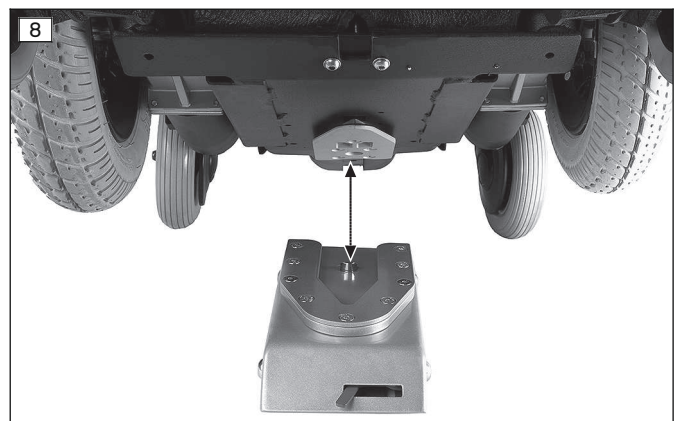
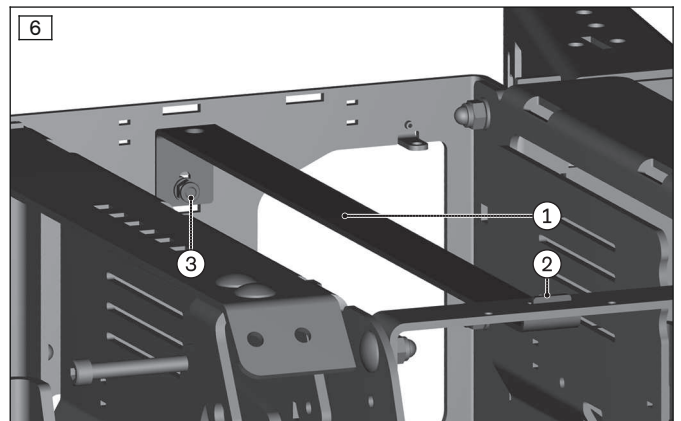
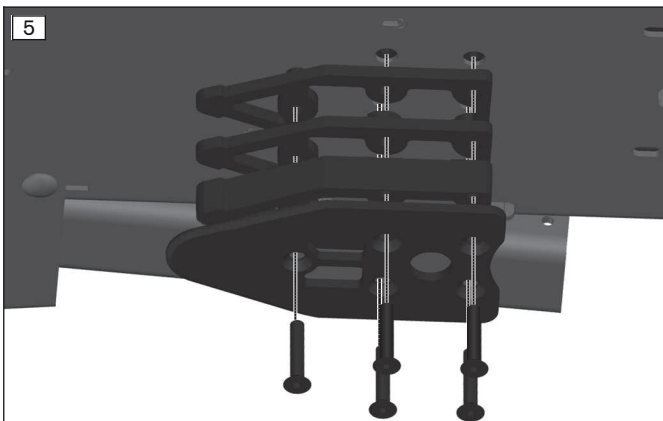
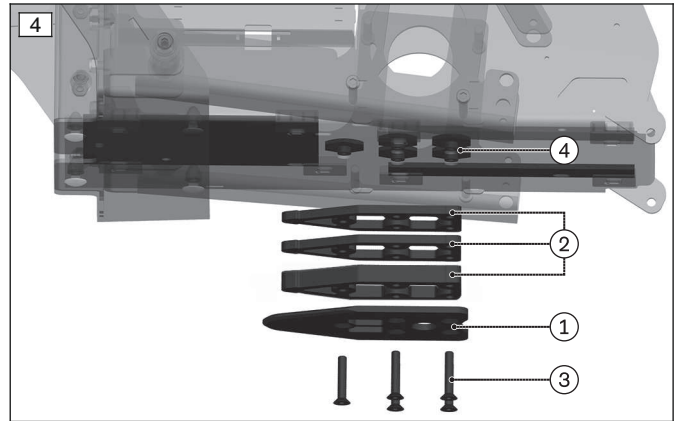
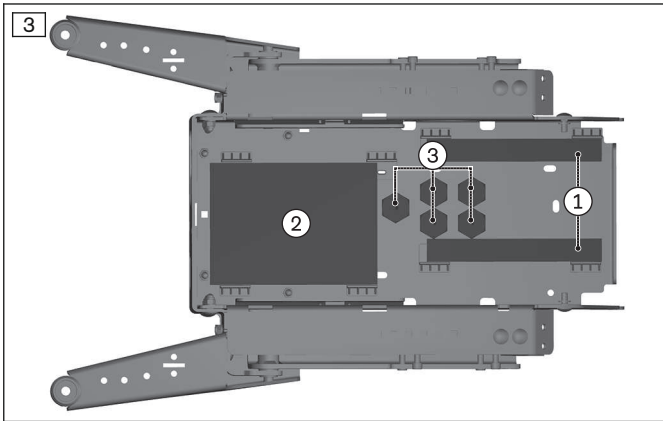
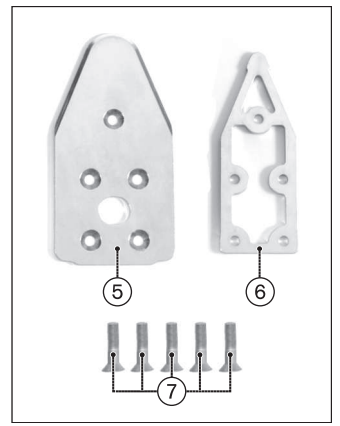
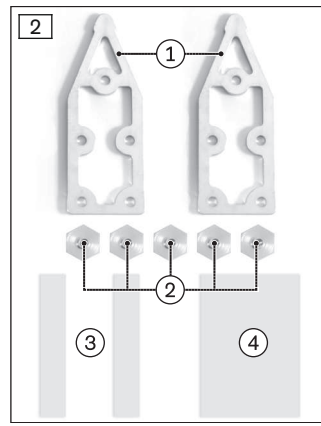
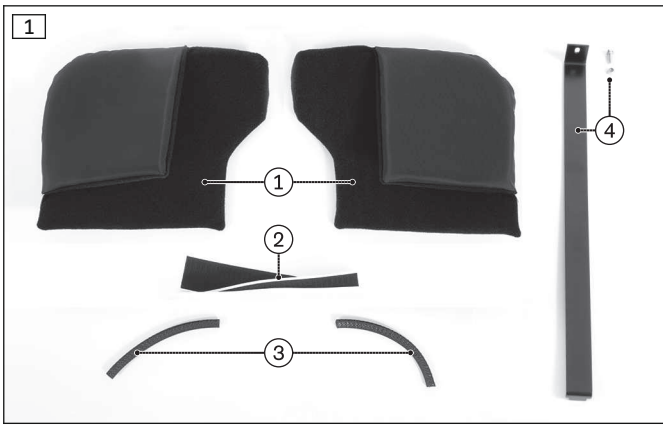
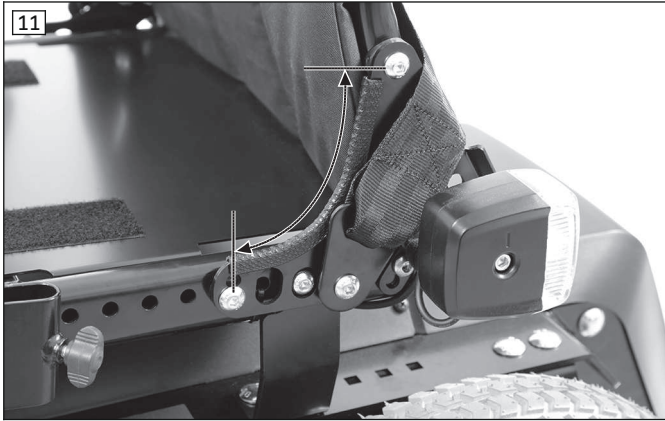
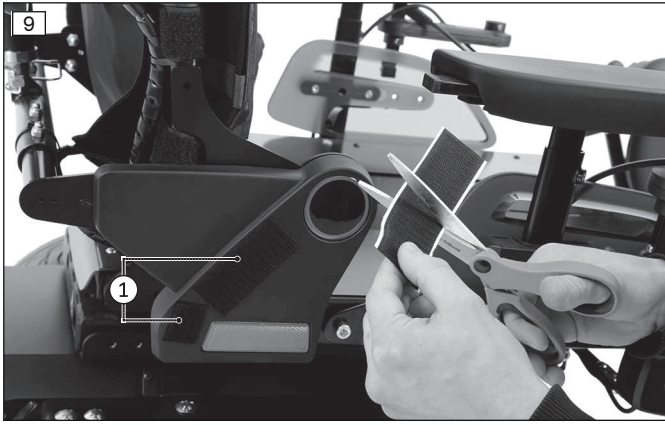
## 491S75=ST610, 491S75=ST615

<b>DE</b> Gebrauchsanweisung .....	5
<b>EN</b> Instructions for use .....	13
<b>FR</b> Instructions d'utilisation .....	21
<b>IT</b> Istruzioni per l'uso .....	29
<b>ES</b> Instrucciones de uso .....	37
<b>PT</b> Manual de utilização .....	45
<b>NL</b> Gebruiksaanwijzing .....	53
<b>SV</b> Bruksanvisning .....	61
<b>DA</b> Brugsanvisning .....	69
<b>NO</b> Bruksanvisning .....	77
<b>FI</b> Käyttöohje .....	85
<b>PL</b> Instrukcja użytkowania .....	93
<b>HU</b> Használati utasítás .....	101
<b>CS</b> Návod k použití .....	109
<b>HR</b> Upute za uporabu .....	117
<b>SL</b> Navodila za uporabo .....	125
<b>SK</b> Návod na používanie .....	133
<b>TR</b> Kullanma talimatı .....	141
<b>RU</b> Руководство по применению .....	149
<b>ZH</b> 使用说明书 .....	159
<b>KO</b> 사용 설명서 .....	167

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DE

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# Table of contents

EN

<b>1</b>	<b>Foreword</b> .....	<b>14</b>
<b>2</b>	<b>Product description</b> .....	<b>14</b>
<b>3</b>	<b>Intended use</b> .....	<b>14</b>
3.1	Area of application .....	14
3.2	Qualification .....	14
<b>4</b>	<b>Safety</b> .....	<b>15</b>
4.1	Explanation of warning symbols .....	15
4.2	Safety Instructions for Assembly .....	15
<b>5</b>	<b>Delivery</b> .....	<b>15</b>
5.1	Scope of delivery .....	15
<b>6</b>	<b>Preparation for use</b> .....	<b>16</b>
6.1	Required tools .....	16
6.2	Installation .....	16
<b>7</b>	<b>Use</b> .....	<b>18</b>
7.1	Prescribed use of head supports .....	18
7.2	Use in vehicles for transporting persons with reduced mobility .....	18
7.3	Cleaning .....	19
<b>8</b>	<b>Maintenance and repair</b> .....	<b>19</b>
8.1	Maintenance .....	19
8.2	Repair .....	19
<b>9</b>	<b>Disposal</b> .....	<b>20</b>
9.1	Disposal information .....	20
<b>10</b>	<b>Legal information</b> .....	<b>20</b>
10.1	Liability .....	20
10.2	Warranty .....	20

# 1 Foreword

**INFORMATION**

Last update: 2018-08-30

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

**INFORMATION**

- ▶ New information regarding product safety and product recalls can be obtained from the Customer Care Center (CCC) at [oa@ottobock.com](mailto:oa@ottobock.com) or from the manufacturer's service department (see inside back cover or back page for addresses).
- ▶ You can request this document as a PDF file from the Customer Care Center (CCC) at [oa@ottobock.com](mailto:oa@ottobock.com) or from the manufacturer's service department (see inside back cover or back page for addresses). It is possible to increase the display size of the PDF document.

These instructions for use provide you with important information on the installation of attachments (protective pads, edge protection, longitudinal bracing for battery mounting), which are required for the use of the "Dahl docking system" option on approved Ottobock power wheelchairs.

They also describe the installation of the locking plate, which is provided by Dahl as a part of the "Dahl docking system" option, on approved Ottobock power wheelchairs.

**Please note the following:**

- Ottobock only provides the attachments required for the use of the Dahl docking system. These are part of the Ottobock installation kit for the "Dahl docking system" option (see Page 15). There, information is also provided on the parts required from Dahl and where these can be ordered.
- Please read these instructions for use prior to installation. The user must also read the instructions before using the system. These instructions for use must be issued to the user and stored in an easy to reach location in the vehicle.

## 2 Product description

**Information on the function of the attachments supplied by Ottobock (protective pads, edge protection)**

The attachments supplied by Ottobock must be installed and used prior to the use of the Dahl docking system, otherwise the belts of the occupant restraint system could become damaged and their function could be adversely affected. The attachments supplied reduce the mechanical forces which act on the belt system of the occupant restraint system.

**Information on the function of the "Dahl docking system" option**

The Dahl docking system makes it possible to attach wheelchairs to the floor of a vehicle (e.g. in vehicles for transporting persons with reduced mobility). For this purpose, the locking plate supplied by Dahl must first be installed under the wheelchair (see the section "Installation"). When the wheelchair is driven to the docking station, it is then attached to the floor of the vehicle using the locking plate. For more detailed information on the overall function of the Dahl docking system, please consult the operating instructions provided by Dahl.

## 3 Intended use

### 3.1 Area of application

The "Dahl docking system" option in conjunction with the attachments supplied by Ottobock was tested in accordance with the ISO 7176-19 standard. It is currently approved for the following Ottobock products:

- 490E75=1\_C

### 3.2 Qualification

The installation and adjustments described below may only be carried out by qualified personnel.

## 4 Safety

### 4.1 Explanation of warning symbols

<b>⚠ WARNING</b>	Warning regarding possible serious risks of accident or injury.
<b>⚠ CAUTION</b>	Warning regarding possible risks of accident or injury.
<b>NOTICE</b>	Warning regarding possible technical damage.

### 4.2 Safety Instructions for Assembly

<b>⚠ WARNING</b>
<b>Uncontrolled movement of components when making adjustments</b>
Crushing, pinching, blows due to non-observance of the maintenance and repair instructions
<ul style="list-style-type: none"> <li>▶ Ensure that body parts, such as hands or head, are never in the danger zone.</li> <li>▶ Perform the work with the aid of a helper for support.</li> </ul>

<b>⚠ CAUTION</b>
<b>Improper lifting</b>
Crushing, pinching, blows due to failure to observe safety instructions
<ul style="list-style-type: none"> <li>▶ Some parts of the power wheelchair, such as the batteries, frame, seat, and motors, are very heavy. Ensure ergonomically correct lifting of these parts. Use sufficiently large hoisting devices or perform this work with a helper.</li> <li>▶ If it is necessary to work underneath raised parts or equipment, make sure these are secured by suitable means so that they cannot come loose, shift, or fall down.</li> <li>▶ When using lifting platforms, ensure that the power wheelchair is centred on the platform and that no parts protrude into the danger zone.</li> </ul>

<b>NOTICE</b>
<b>Improper preparation before making adjustments</b>
Damage to the product due to falling down and failure to follow setting instructions
<ul style="list-style-type: none"> <li>▶ When you work on the product, secure it so that it cannot tip over or fall down.</li> <li>▶ Turn the power wheelchair off and deactivate the circuit breaker before making any adjustments. Functional tests of the electrical components are excepted from this rule.</li> <li>▶ Before performing any work on the seat, ensure that the cushion is sufficiently protected against mechanical, chemical, and thermal effects.</li> </ul>

## 5 Delivery

### 5.1 Scope of delivery

The Ottobock scope of delivery includes:

**Ottobock installation kit for the "Dahl docking system"** (see fig. 1)

<b>Order number</b>	490E75=1_C: 491S75=ST610 (standard seat); 491S75=ST615 (VAS seat)	
Item	Article designation	Quantity
1	Protective pads (for VAS seat)	2
2	Hook-and-loop strips, self-adhesive (for VAS seat)	2
3	Edge protection (for standard seat)	2
4	Longitudinal bracing for battery mounting (including mounting materials)	1

The following parts are required from Dahl:

**Adaption set supplied by Dahl for 490E75=1\_C/Ottobock\*** (see fig. 2, left)

<b>Order number</b>	503184*	
Item	Article designation	Quantity

Item	Article designation	Quantity
1	Intermediate piece, 2 mm	2
2	M8 hexagon nut	5
3	Spacer strip, PVC, 3 mm	2
4	Spacer plate, PVC, 3 mm	1

### Parts from the basic kit supplied by Dahl (see fig. 2, right)

<b>Order number</b>	501750*
---------------------	---------

Item	Article designation	Quantity
5	Locking plate (500561*)	1
6	Intermediate piece, 8 mm (500673*)	1
7	Mounting screws M8x45, Torx (502800*)	5

\* Listed Dahl order number. Order via: Dahl Engineering ApS | Løvevej 3 | DK-7700 Thisted | Denmark | Tel: +45 96 18 00 77 | sales@dahlengineering.dk | http://wtors.com

## 6 Preparation for use

### 6.1 Required tools

The following tools are required for adjustments and maintenance work:

#### INFORMATION

The following list only includes tools which are required for the direct assembly of the parts specified in the scope of supply of Dahl and Ottobock. Please refer to the service manual for information on tools for preparatory work (e.g. for the removal of the battery).

- Allen keys, size 4
- Ring and open-ended spanner, size 8, 24
- Torque wrench (measurement ranges 5-50 Nm)
- Torx screwdriver, size T45
- Metal saw
- Scissors or cutter knife
- Liquid thread lock, "medium strength" (e.g. Loctite 241)

### 6.2 Installation

#### Safety instructions for the installation of the Dahl parts on the Ottobock power wheelchair

#### ⚠ WARNING

##### Non-observance of the Dahl user guide

Severe injury in case of accidents due to installation errors

- ▶ The installation and safety instructions supplied by Dahl, which are included with the installation and operating instructions for the Dahl docking system, must be observed during all installation work.
- ▶ The following installation notes are only valid in conjunction with the installation and operating instructions supplied by Dahl.

#### ⚠ CAUTION

##### Incorrect use of mounting screws

Severe injuries due to breakage of the Torx mounting screws

- ▶ Only use the Torx mounting screws supplied by Dahl Engineering. Other standard screws are not strong enough to hold the attachment plate in case of a collision.
- ▶ Please note that the Torx mounting screws have to be shortened during installation. Do not damage the thread when shortening the screws.
- ▶ Please note that the Torx mounting screws have to be secured with "medium strength" liquid thread lock (e.g. Loctite 241) during installation.

**INFORMATION**

Please note that the following steps refer in part to instructions in the service manual. You can request the PDF file of the service manual from the Customer Care Center (CCC) at [oa@ottobock.com](mailto:oa@ottobock.com) or from the manufacturer's service department (see inside back cover or back page for addresses). Before starting the installation work, ensure that the service manual is available:

- ▶ Service manual for 490E75=1\_C; 490E75=1\_Z: 647G1999

**Installation of the Dahl attachment plate on the drive support**

- 1) Prepare access to the drive support:
  - **In the equipment version without power seat functions:** Remove the seat cushion and disassemble the seat plate.
  - **In the equipment version with power seat functions:** Move the seat away from the battery cover as far as possible.
- 2) Switch off the control device and remove the main fuse.
- 3) Slightly jack up the power wheelchair (see service manual).
- 4) Open the battery cover (see service manual). Disassemble the transverse and longitudinal bracing (see service manual).
 

**INFORMATION: Please note that, to complete the work, the longitudinal bracing has to be replaced with the newly supplied longitudinal bracing from the Ottobock installation kit for the Dahl docking system.**
- 5) Remove the batteries (see instructions for use for qualified personnel or service manual). The floor of the drive support is now freely accessible.
- 6) Place the spacer strips on the floor of the drive support in accordance with the illustration (see fig. 3, item 1).
- 7) Place the spacer plate on the floor of the drive support in accordance with the illustration (see fig. 3, item 2).
- 8) Have the 5 hexagon screws ready for installation at a later stage (see fig. 3, item 3).
- 9) Have the locking plate (see fig. 4, item 1) and the 3 matching intermediate pieces (see fig. 4, item 2) ready. Also have the 5 Torx mounting screws ready for this purpose (see fig. 4, item 3).
- 10) Insert the Torx mounting screws one after the other from the bottom through the locking plate, the 3 intermediate pieces and the 5 holes of the drive support (see fig. 5).
- 11) Provisionally screw on the Torx mounting screws (see fig. 4, item 3) to the drive support from the top, using the 5 hexagon nuts (see fig. 4, item 4).
- 12) Mark the excess. Remove the screws, shorten them and carefully deburr the top.
 

**WARNING! The screws must be flush with the hexagon nut. They may neither protrude beyond the hexagon nuts or be too short to achieve their full holding force.**
- 13) Re-insert the Torx mounting screws from the bottom and secure them with "medium strength" liquid thread lock (e.g. Loctite 241).
- 14) Screw on the 5 hexagon nuts onto the Torx mounting screws from the top and tighten with **18 Nm** (see fig. 4, item 3/4).
- 15) Re-insert the batteries. Install the transverse bracing (see service manual).
- 16) Install the new longitudinal bracing (see fig. 6, item 1). To do so, fit the longitudinal bracing in the transverse bracing (see fig. 6, item 2) and mount the longitudinal bracing on the drive support using the hexagon socket screw, the washer and the hexagon nut (see fig. 6, item 3). The washer and the hexagon nut are positioned inside the drive support.
- 17) Close the battery cover again (see service manual).
  - The locking plate of the Dahl docking system is installed under the power wheelchair (see fig. 7).

**Installation of the attachments supplied by Ottobock on the VAS seat (protective pads)**

- 1) Cut the hook-and-loop strap to size. Select the length so that the space between the pivot point and the reflector is well filled (see fig. 9, item 1).
- 2) Stick the hook-and-loop strap to the cover diagonally below the pivot point.
- 3) Guide the protective pad over the cover and attach it to the hook-and-loop strap (see fig. 10).
- 4) Attach the second protective pad on the other side in the same way.

**Installation of the attachments supplied by Ottobock on the standard seat (edge protection)**

- 1) Cut the edge protection to size if required. The length is prescribed by the length of the edge of the end plate (see fig. 11, arrow).
- 2) Position the edge protection on the edge of the end plate and press it on by hand (see fig. 12).

- 3) Attach the second edge protection on the other side in the same way.

## 7 Use

### 7.1 Prescribed use of head supports

#### **⚠ WARNING**

##### **Non-observance of the Ottobock user guide**

Serious injuries in case of accidents due to user error

- ▶ When using the Dahl docking system with approved Ottobock power wheelchairs, the installation of an Ottobock head support approved for use in combination with the Dahl docking system is mandatory.
- ▶ Note the list of possible head supports below.

The following head supports are approved for use in connection with the Dahl docking system:

Code number	Description	Article number
EL006	Head support, small	430H1=3-7
EL007	Head support, large	430H1=1-7
EL009	Head support, concave, size M	476L49=SK002
EL011	Head support with ABS exterior shell, small	430H2=3-7
EL012	Head support with ABS exterior shell, large	430H2=1-7
EL013	Head/neck support, small	430C1=3-7
EL014	Head/neck support, large	430C1=1-7
EL015	Head/neck support, X-shape, small	430C4=1-7
EL016	Head/neck support, X-shape, large	430C3=1-7
EL017	Head support, concave, adjustable, one size only	476L49=SK020
EL020	Head support, 3-piece, angle-adjustable, size L	476L49=SK012

### 7.2 Use in vehicles for transporting persons with reduced mobility

#### **⚠ WARNING**

##### **Non-observance of the Dahl user guide**

Serious injuries in case of accidents due to user error

- ▶ Before using the Dahl docking system in vehicles for transporting persons with reduced mobility, read the installation and operating instructions and in particular the safety notices of the manufacturer.
- ▶ Only use the Dahl docking system if the attachments supplied by Ottobock (protective pads, edge protection) are installed on the power wheelchair. Otherwise the belts of the occupant restraint system could become damaged and their function could be adversely affected.

#### **⚠ WARNING**

##### **Positioning in vehicles for transporting persons with reduced mobility**

Serious injuries in case of accidents due to user error

- ▶ The positioning of the product in vehicles for transporting persons with reduced mobility may only be performed by qualified personnel.
- ▶ The product must always face forwards when it is used as a seat in a vehicle for transporting persons with reduced mobility.
- ▶ Please observe the positioning information in the instructions for use supplied by the manufacturer Dahl.

#### **⚠ WARNING**

##### **Use of the belt system as a passenger restraint system in vehicles for transporting persons with reduced mobility is forbidden**

Serious injuries due to improper handling of the product

- ▶ Under no circumstances may the belts and positioning aids offered with the product be used as part of a passenger restraint system in vehicles for transporting persons with reduced mobility.
- ▶ Note that the belts and positioning aids offered with the product are only intended to help support the user sitting in the product.

**⚠ WARNING****Using the product with certain settings and/or installed accessories**

Severe injury in case of accidents due to accessories coming loose

- ▶ Before using the product as a seat in a vehicle for transporting persons with reduced mobility, remove accessories that need to be dismantled for safe transportation in a vehicle for transporting persons with reduced mobility. Observe the information on the main product in the instructions for use.
- ▶ Stow all dismantled accessories securely in the vehicle for transporting persons with reduced mobility.

Prior to the use of the "Dahl docking system" in a vehicle for transporting persons with reduced mobility, it must be ensured that the protective aids supplied by Ottobock are attached to the seat of the power wheelchair (protective pads for VAS seat: see fig. 10; edge protection for standard seat: see fig. 11). This is the only way to ensure that the occupant restraint belts are not damaged by sharp corners in case of a crash.

For the use of the power wheelchair, attention must also be paid to the information provided in the instructions for use for the main Ottobock product, in particular the section "Use in vehicles for transporting persons with reduced mobility".

For the use of the Dahl docking system in vehicles for transporting persons with reduced mobility, attention must be paid to the instructions for use in the operating instructions by the manufacturer, Dahl. This applies in particular to the following contents:

- Attachment of the wheelchair in the docking station
- Release from the docking station

**7.3 Cleaning****Cleaning the attachments supplied by Ottobock (protective pads, edge protection)**

- The pads can be machine-washed in warm water up to **max. 60 °C (140 °F)**, using a mild detergent.
- Clean the hook-and-loop closure with a dry brush.
- Allow the product to fully air dry before use.
- Do not expose the product to direct heat (e.g. sunshine, stove or radiator heat, hair dryer).

**Cleaning the parts of the Dahl docking system**

For cleaning the parts of the Dahl docking system, attention must be paid to the information in the operating instructions by the manufacturer, Dahl.

**8 Maintenance and repair****8.1 Maintenance****Maintenance instructions for the attachments supplied by Ottobock (protective pads, edge protection)**

- Maintenance is not required if the attachments are cleaned regularly.

**Maintenance instructions for the "Dahl docking system" option**

- For correct maintenance, attention must be paid to the information in the operating instructions by the manufacturer, Dahl.
- The function of the product should be checked **before each use**.
- The product may not be used if defects are noted. This applies in particular in case of instability of the product or altered driving characteristics as well as problems with the user's seating position or the stability of the seat. Inform the qualified personnel promptly for the rectification of defects.
- This also applies if loose, worn, bent or damaged components, cracks in the frame or broken frame components are identified.
- Failure to maintain the product can lead to injuries for the user of the product.

**8.2 Repair****Repair instructions for the attachments supplied by Ottobock (protective pads, edge protection)**

- In case of damage, promptly contact the Ottobock service department (see inside back cover or back page for the address).

**Repair instructions for the "Dahl docking system" option**

- In case of damage, promptly contact the qualified personnel who adapted the product or the manufacturer's service department (for the address, see the enclosed operating instructions of the manufacturer, Dahl).

## **9 Disposal**

### **9.1 Disposal information**

Return the product to the specialist dealer for disposal.

All components of the product must be disposed of properly in accordance with the respective national environmental regulations.

## **10 Legal information**

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### **10.1 Liability**

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### **10.2 Warranty**

Further information on the warranty terms and conditions can be obtained from the qualified personnel that has fitted this product or the manufacturer's service (see inside back cover for addresses).

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A series of horizontal lines spanning the width of the page, providing a template for writing or drawing.

# Kundenservice/Customer Service

## Europe

Otto Bock HealthCare Deutschland GmbH  
Max-Näder-Str. 15 · 37115 Duderstadt · Germany  
T +49 5527 848-3433 · F +49 5527 848-1460  
healthcare@ottobock.de · www.ottobock.de

Otto Bock Healthcare Products GmbH  
Brehmstraße 16 · 1110 Wien · Austria  
F +43 1 5267985  
service-admin.vienna@ottobock.com · www.ottobock.at

Otto Bock Adria d.o.o. Sarajevo  
Ramiza Salčina 85  
71000 Sarajevo · Bosnia-Herzegovina  
T +387 33 255-405 · F +387 33 255-401  
obadria@bih.net.ba · www.ottobockadria.com.ba

Otto Bock Bulgaria Ltd.  
41 Tzar Boris III Blvd. · 1612 Sofia · Bulgaria  
T +359 2 80 57 980 · F +359 2 80 57 982  
info@ottobock.bg · www.ottobock.bg

Otto Bock Suisse AG  
Luzerner Kantonsspital 10 · 6000 Luzern 16 · Suisse  
T +41 41 455 61 71 · F +41 41 455 61 70  
suisse@ottobock.com · www.ottobock.ch

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Otto Bock South Africa (Pty) Ltd  
Building 3 Thornhill Office Park · 94 Bekker Road  
Midrand · Johannesburg · South Africa  
T +27 11 564 9360  
info-southafrica@ottobock.co.za  
www.ottobock.co.za

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Otto Bock HealthCare Canada  
5470 Harvester Road  
Burlington, Ontario, L7L 5N5, Canada  
T +1 800 665 3327 · F +1 800 463 3659  
CACustomerService@ottobock.com  
www.ottobock.ca

Oficina Ottobock Habana  
Calle 3ra entre 78 y 80.  
Edificio Jerusalem · Oficina 112 · Calle 3ra.  
Playa, La Habana. Cuba  
T +53 720 430 69 · +53 720 430 81  
hector.corcho@ottobock.com.br  
www.ottobock.com.br

Otto Bock HealthCare Andina Ltda.  
Calle 138 No 53-38 · Bogotá · Colombia  
T +57 1 8619988 · F +57 1 8619977  
info@ottobock.com.co · www.ottobock.com.co

Otto Bock de Mexico S.A. de C.V.  
Prolongación Calle 18 No. 178-A  
Col. San Pedro de los Pinos  
C.P. 01180 México, D.F. · Mexico  
T +52 55 5575 0290 · F +52 55 5575 0234  
info@ottobock.com.mx · www.ottobock.com.mx

Otto Bock HealthCare LP  
11501 Alterra Parkway Suite 600  
Austin, TX 78758 · USA  
T +1 800 328 4058 · F +1 800 962 2549  
USCustomerService@ottobock.com  
www.ottobockus.com

## Asia/Pacific

Otto Bock Australia Pty. Ltd.  
Suite 1.01, Century Corporate Centre  
62 Norwest Boulevard  
Baulkham Hills NSW 2153 · Australia  
T +61 2 8818 2800 · F +61 2 8814 4500  
healthcare@ottobock.com.au · www.ottobock.com.au

Beijing Otto Bock Orthopaedic Industries Co., Ltd.  
B12E, Universal Business Park  
10 Jiuxianqiao Road, Chao Yang District  
Beijing, 100015, P.R. China  
T +8610 8598 6880 · F +8610 8598 0040  
news-service@ottobock.com.cn  
www.ottobock.com.cn

Otto Bock Asia Pacific Ltd.  
Unit 1004, 10/F, Greenfield Tower, Concordia Plaza  
1 Science Museum Road, Tsim Sha Tsui  
Kowloon, Hong Kong · China  
T +852 2598 9772 · F +852 2598 7886  
info@ottobock.com.hk · www.ottobock.com

Otto Bock HealthCare India Pvt. Ltd.  
20th Floor, Express Towers  
Nariman Point, Mumbai 400 021 · India  
T +91 22 2274 5500 / 5501 / 5502  
information@indiaottobock.com · www.ottobock.in

Otto Bock Japan K. K.  
Yokogawa Building 8F, 4-4-44 Shibaura  
Minato-ku, Tokyo, 108-0023 · Japan  
T +81 3 3798-2111 · F +81 3 3798-2112  
ottobock@ottobock.co.jp · www.ottobock.co.jp

Otto Bock Korea HealthCare Inc.  
4F Agaworld Building · 1357-74, Seocho-dong  
Seocho-ku, 137-070 Seoul · Korea  
T +82 2 577-3831 · F +82 2 577-3828  
info@ottobockkorea.com · www.ottobockkorea.com

Otto Bock South East Asia Co., Ltd.  
1741 Phaholyothin Road  
Kwaeng Chatuchark · Khet Chatuchark  
Bangkok 10900 · Thailand  
T +66 2 930 3030 · F +66 2 930 3311  
obsea@ottobock.co.th · www.ottobock.co.th

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