30.08.2022

Monteringsanvisning / Fitting Instruction

BALDER

- Finesse
- Junior
- Liberty

Dahl Docking System, kompatibel med Docking Station MKII (501750) og VarioDock (503600)

Dahl Docking System, compatible with Docking Station MKII (501750) and VarioDock (503600)

HP20071412

Rullestolfeste, Stodel, inkludert låseplate, 8mm spacer og 14,9 bolter, F/J


Clamp down unit, wheelchair unit, incl. lock plate, 8mm spacer and 14,9 bolts, F/J

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
Clamp down unit, wheelchair unit, not incl. lock plate, 8mm spacer and 14,9 bolts, F/J

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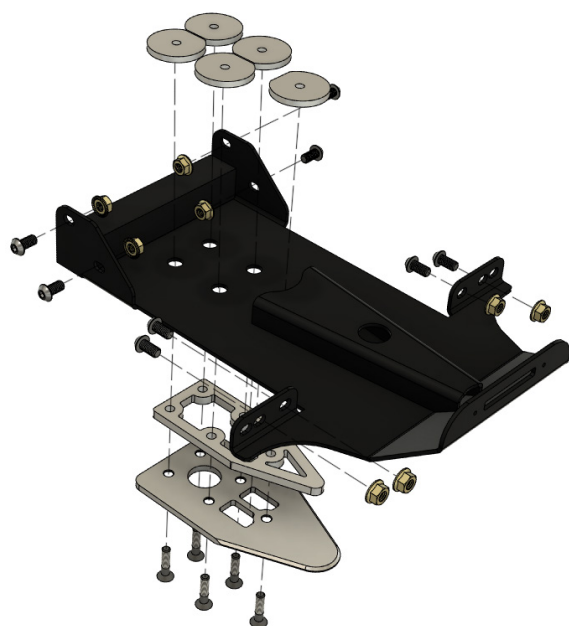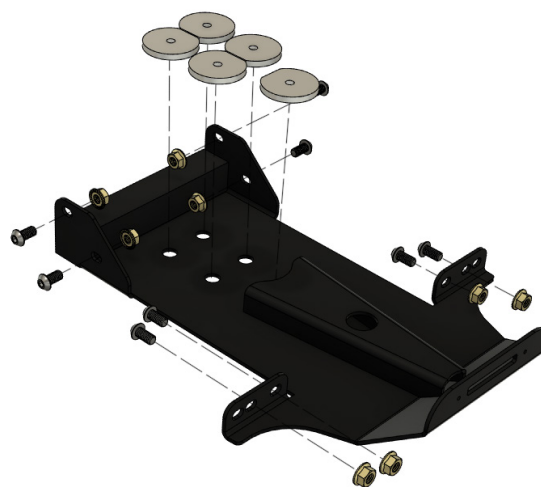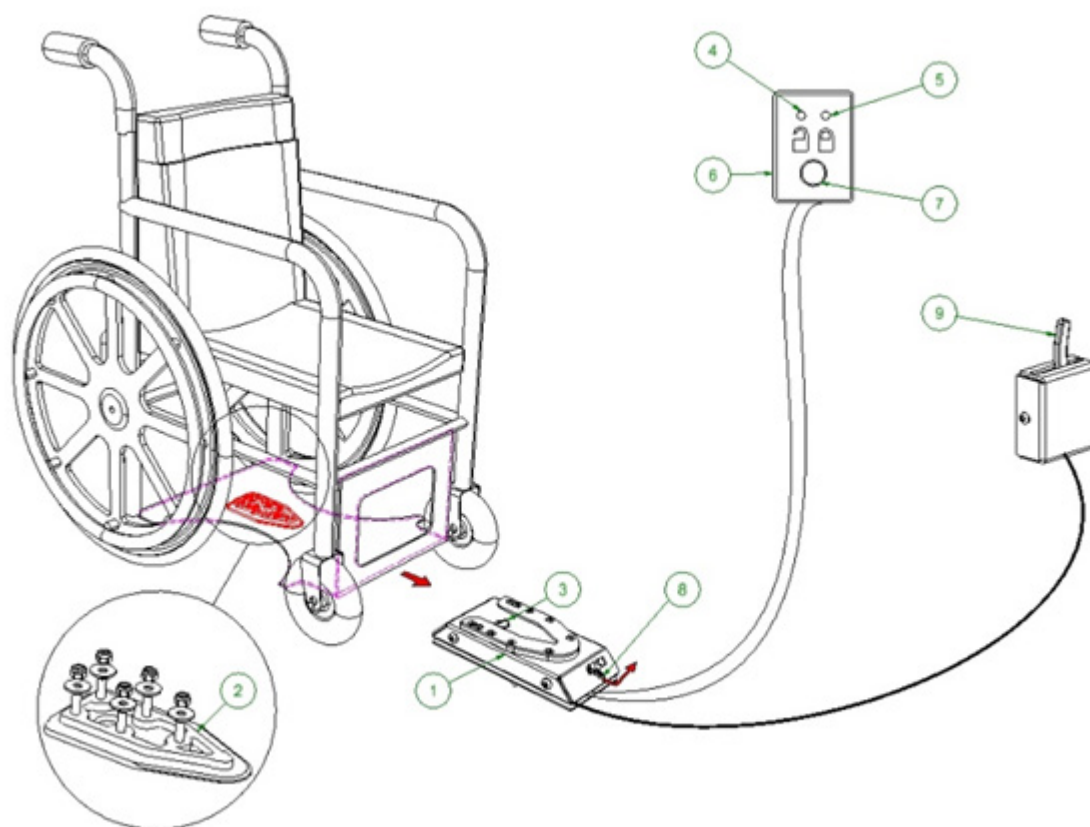
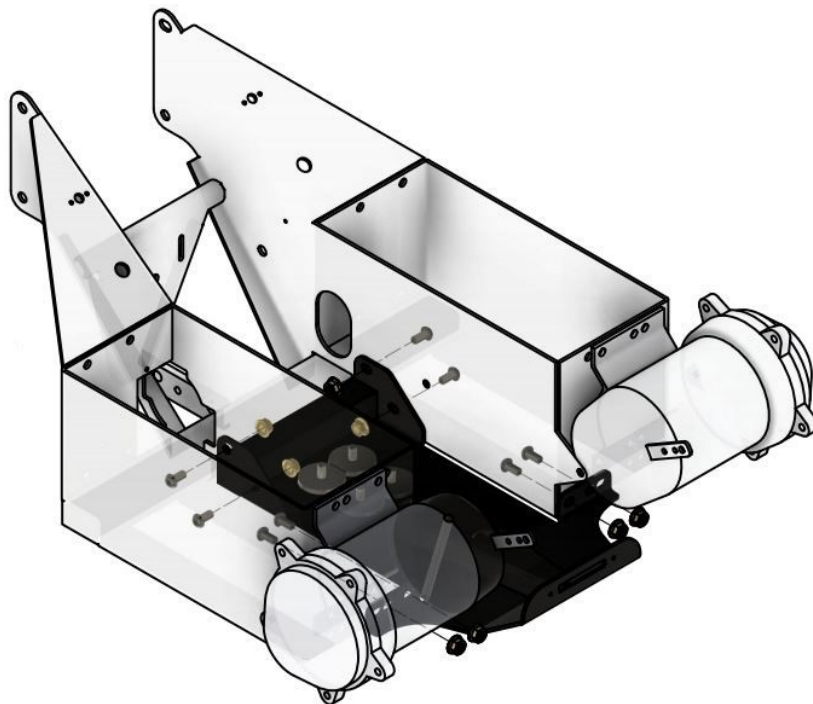
Stolfestet er godkjent i.h.t ISO 7176/19 og 10542

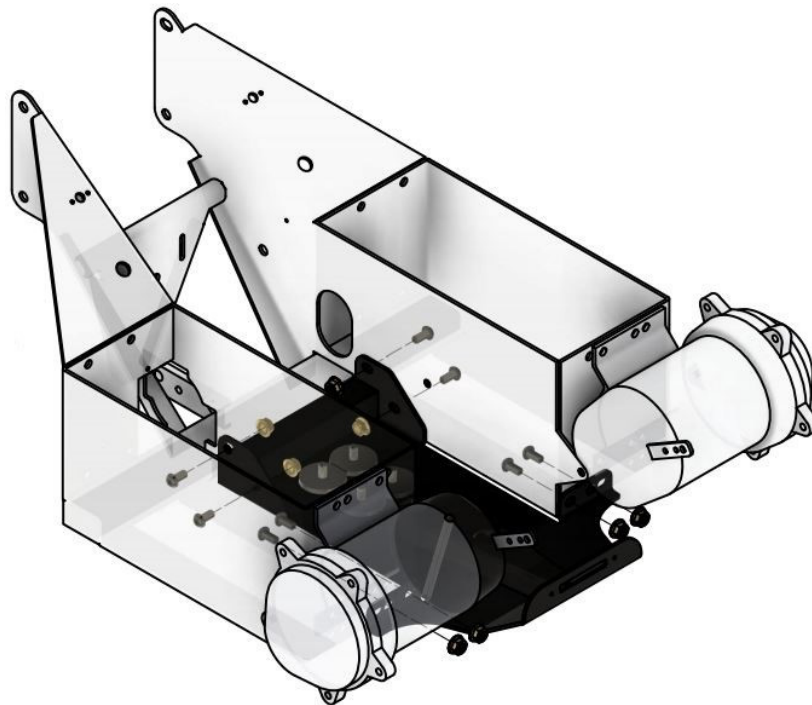
EN**Assembly**

1. Remove the main cover and disconnect the batteries. The batteries must be removed from the chassis.
2. The chair part shall be mounted with eight M8x16 umbraco lens screws and eight M8dur-lock nuts. The chair part shall be placed horizontally to the lower edge of the wheelchair's chassis.
3. The lens screws should be inserted from the inside of the battery box and out towards the chair part to prevent damage to the batteries.
4. Place nuts in predrilled holes in the chair part with the recess down.
5. Fit the 8 mm spacer onto lock plate and put the five bolts (Dahl #502800) through the lock plate and 8 mm spacer.
6. Mount the bolts in the nuts and tighten to a torque of 16-18 Nm.

WARNING!

Do not use any other bolts than those supplied from Dahl Engineering (part#502800 which is quality 14.9, torx key size 27). Standard countersunk M8 bolts will not be strong enough in the event of a collision.

7. Apply loctite 222 (or an equivalent product) onto threads on all bolts.
8. Set the batteries back into their position, assemble them and put the cover back on again.
9. If the chair has wheels filled with air, it is important to make sure they have the correct air pressure: 30 PSI in the big wheels and 50 PSI.
10. Perform final check by connecting the wheelchair to the docking station. Make sure that lock plate is securely locked and that all release methods work as intended. A warning tone will sound if lock plate is not properly engaged.

**Approval**

The clamp down unit is approved according to ISO 7176/19 and 10542.