User manual

Kudu

R82[®]
by Etac

M1015 - 2025-09-08 - Rev. 25

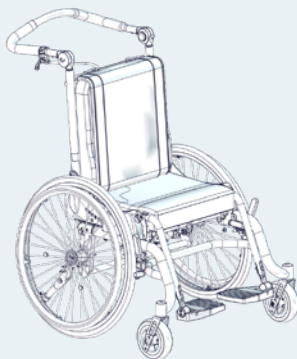
www.etac.com

MD


CE

UK
CA


	4		31
	5-6		32
	7-8		33
	9-10		34-39
	11	mm (inch) kg (lb)	40-41
	12		42-44
	13 - 15	ENGLISH (Original instructions) DEUTSCH	45-50 51-57
	16	NEDERLANDS DANSK	58-64 65-70
	17	NORSK SVENSKA	71-76 77-82
	18 - 19	SUOMI ESPAÑOL	83-88 89-95
	20	FRANÇAIS ITALIANO	96-102 103-109
	21	PORTUGUÊS PORTUGUÊS DO BRASIL	110-116 117-123
	22 - 23	РОССИЯ POLSKI	124-130 131-137
	24 - 25	ČESKY 中國	138-143 144-149
	26 - 27	日本 ΕΛΛΗΝΙΚΗ	150-156 157-163
	28-29	ÍSLENSKA	164-169
	30		


KUDU 1

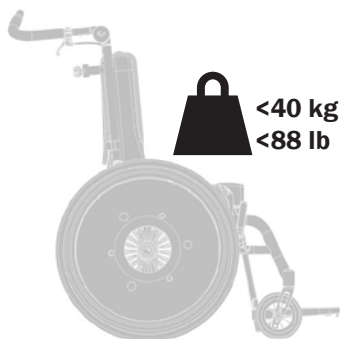
KUDU 2






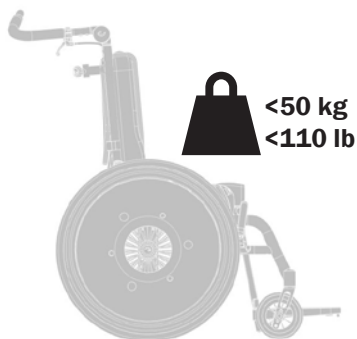
KUDU 3








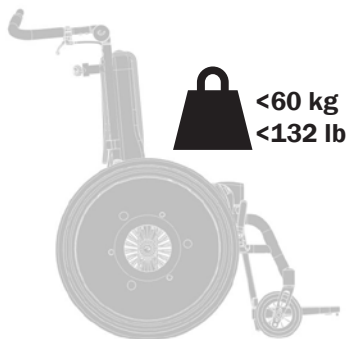
KUDU 4

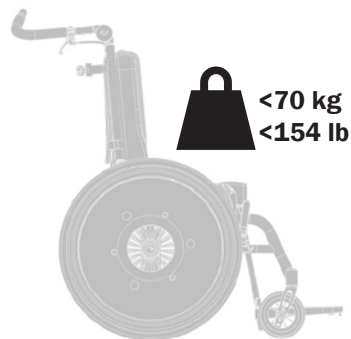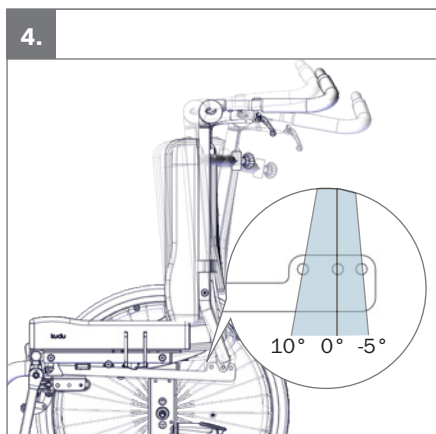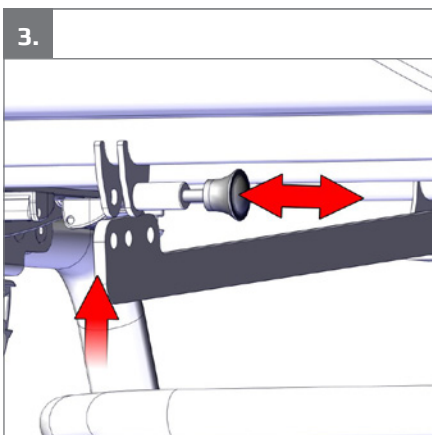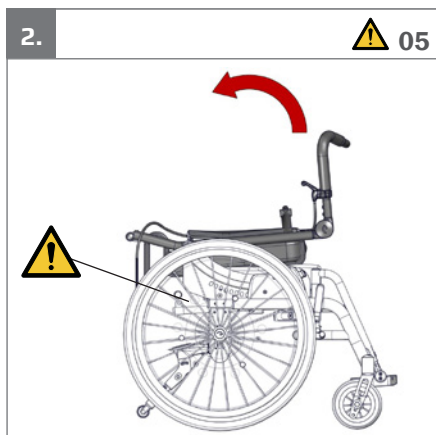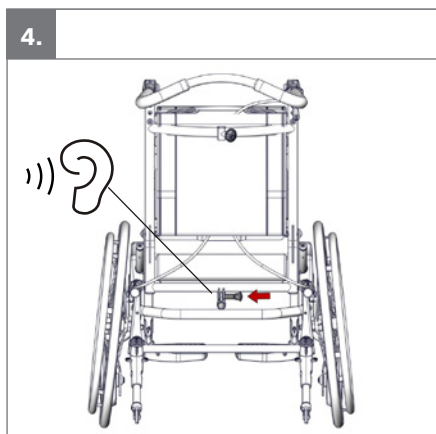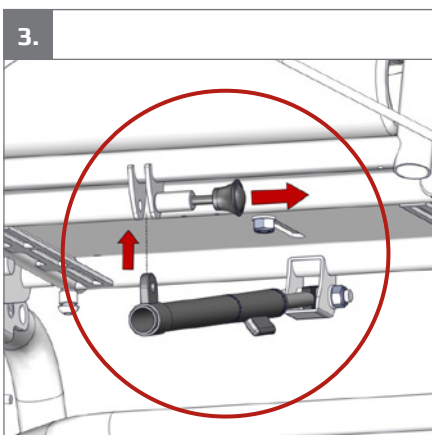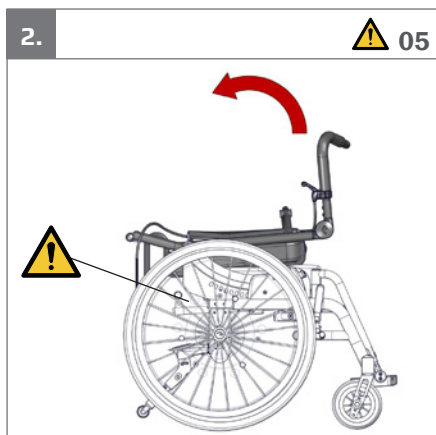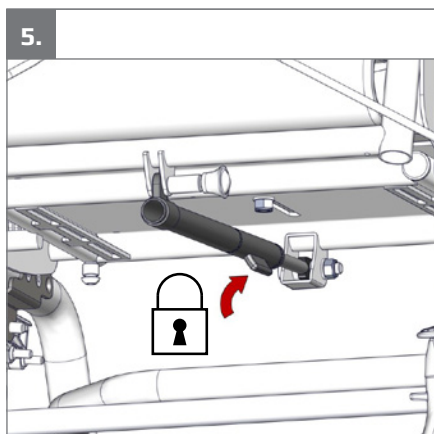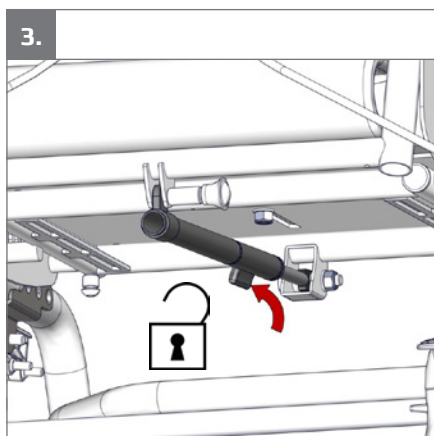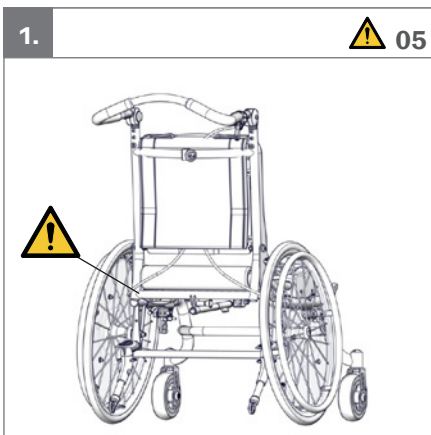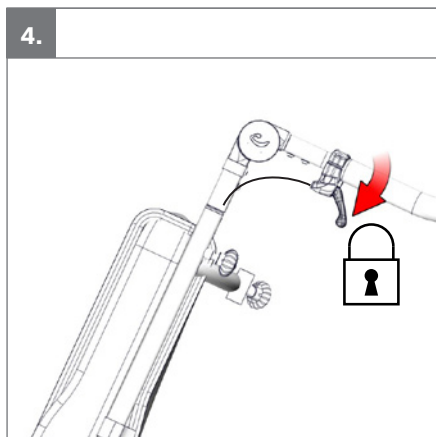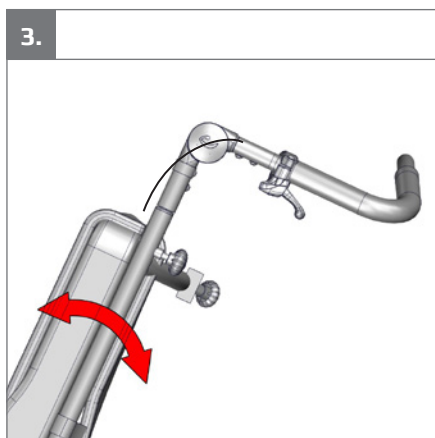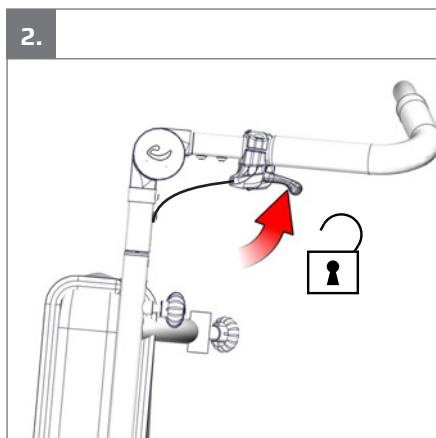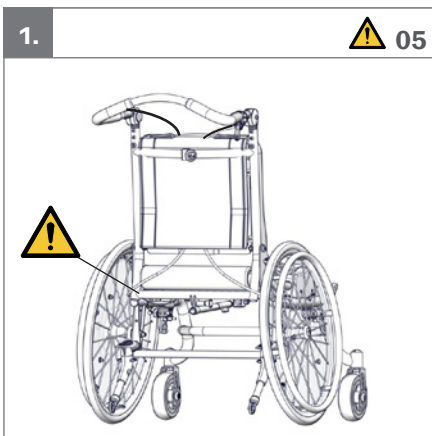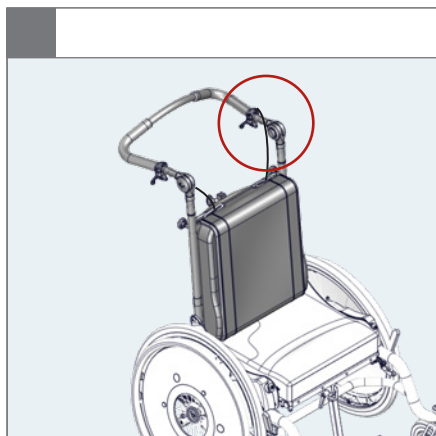
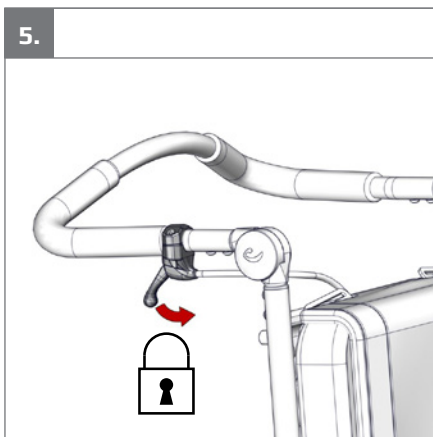
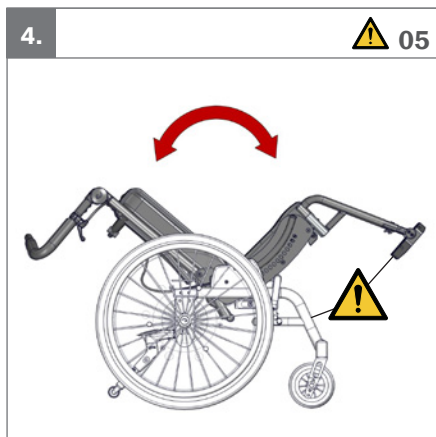
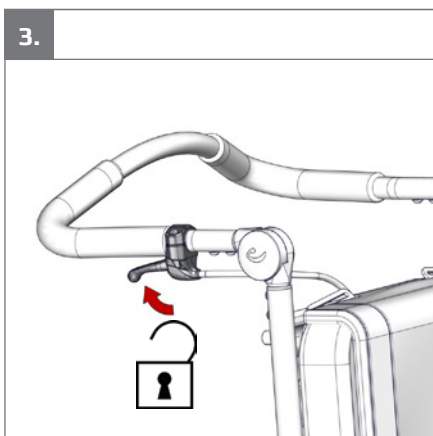
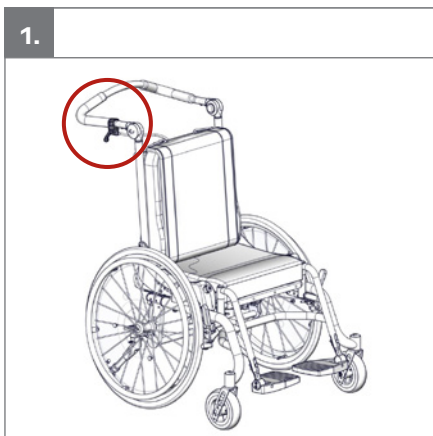
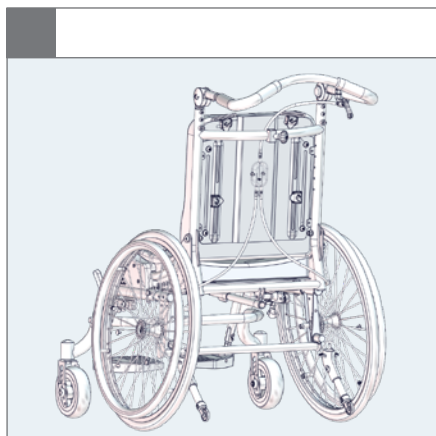
1.

⚠ 05

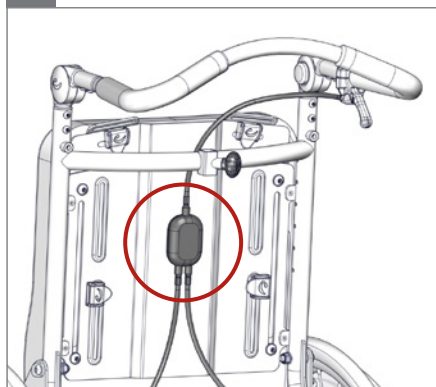
2.


3.

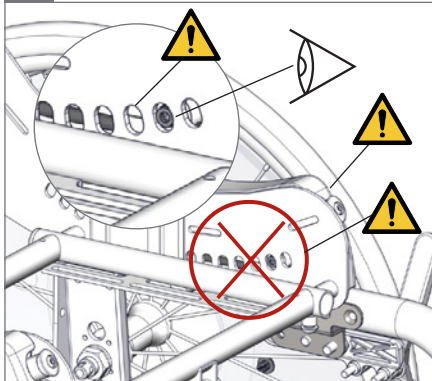
⚠ 01 & 05











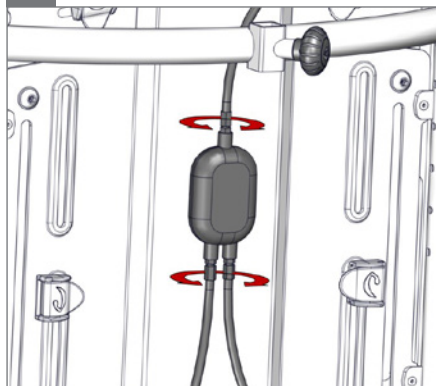
4.

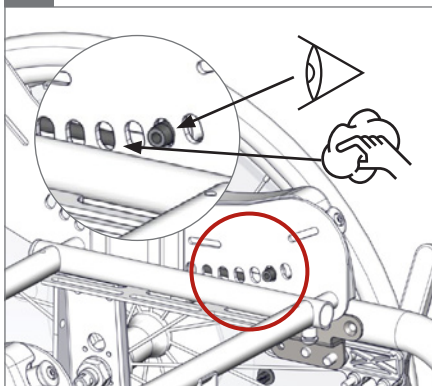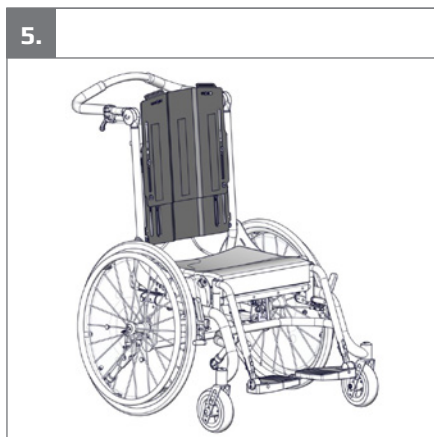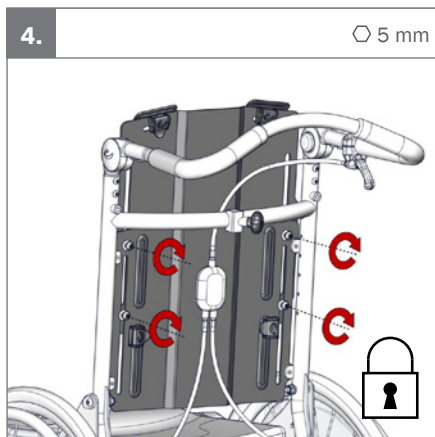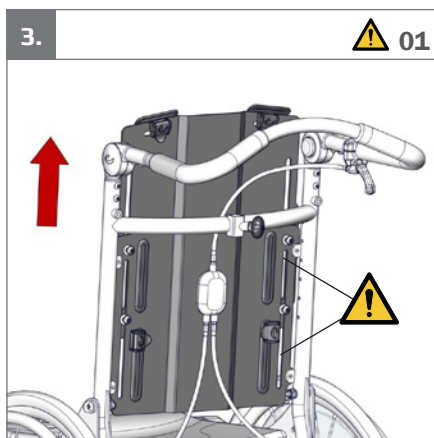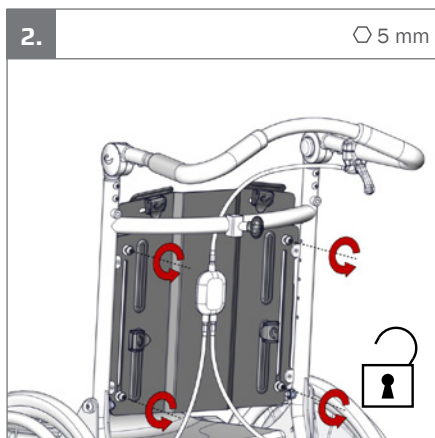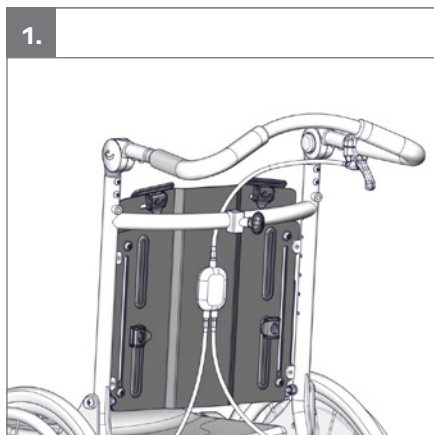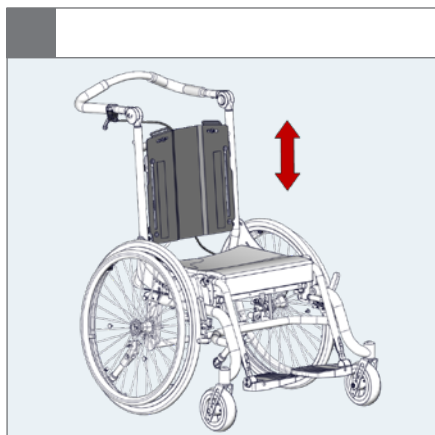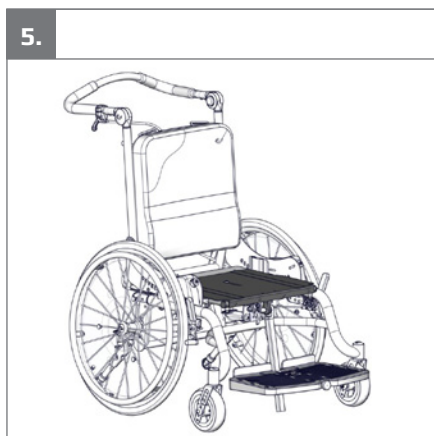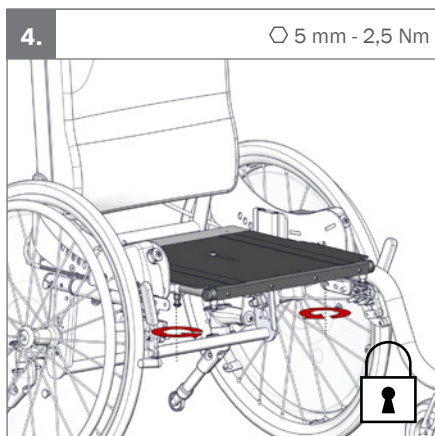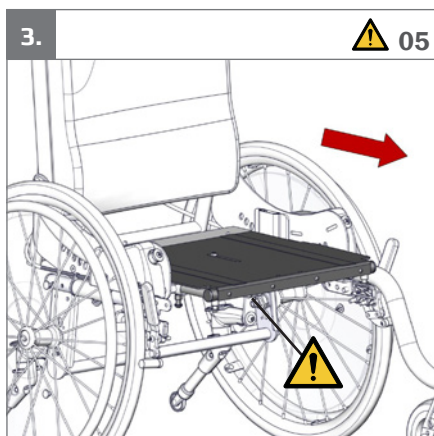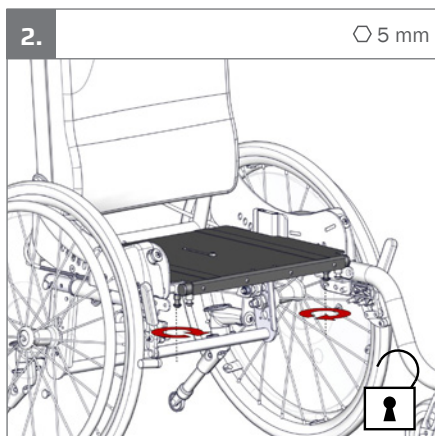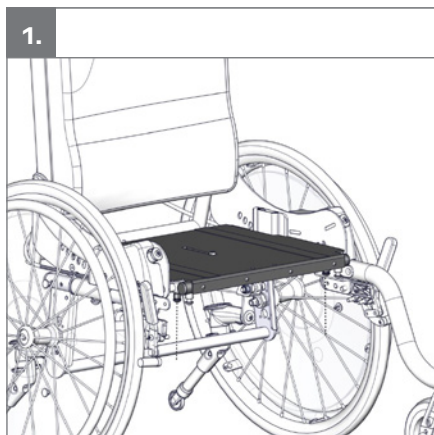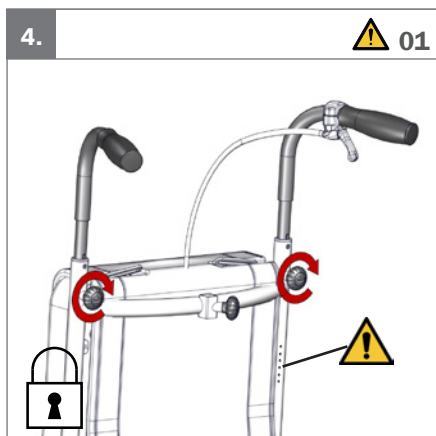
5.


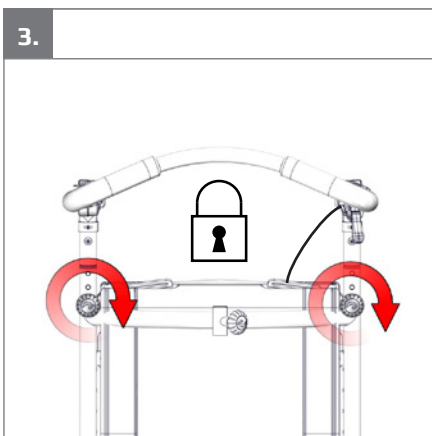
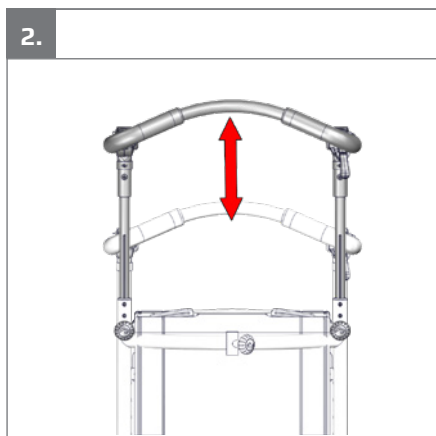
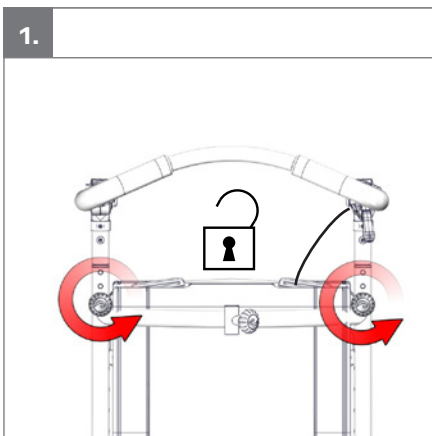
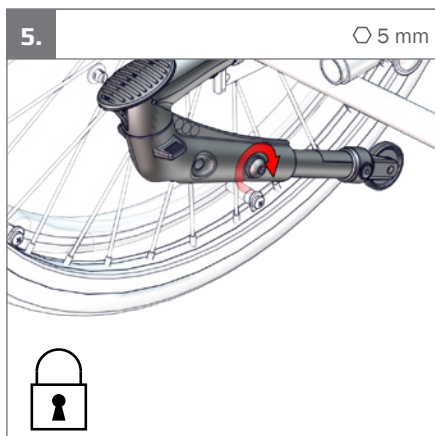
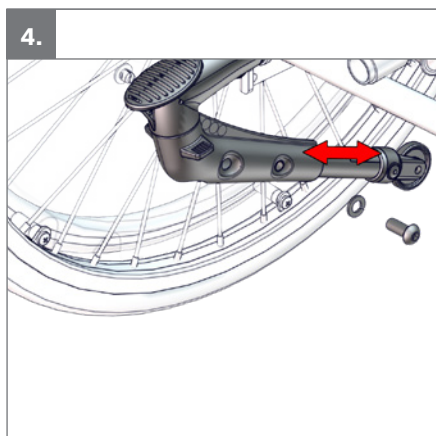
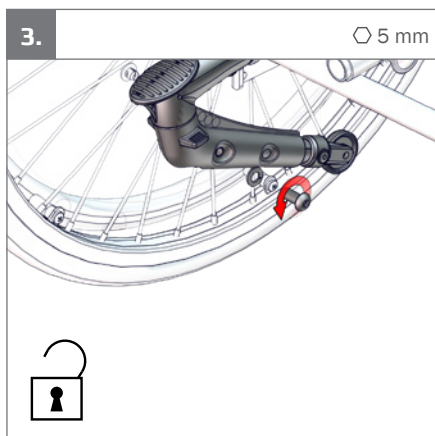
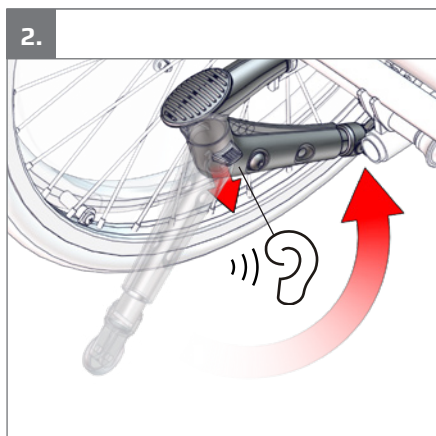
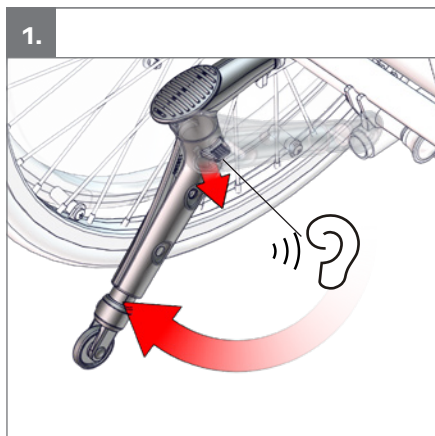
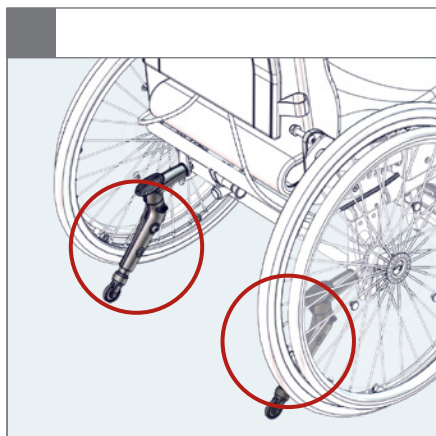
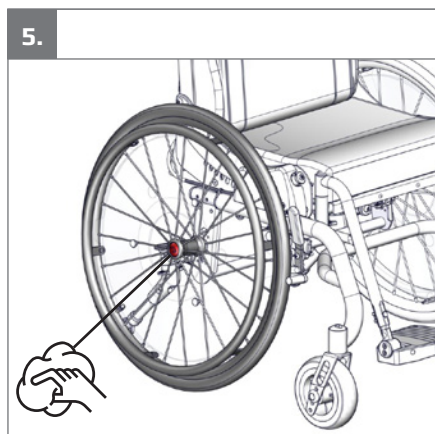
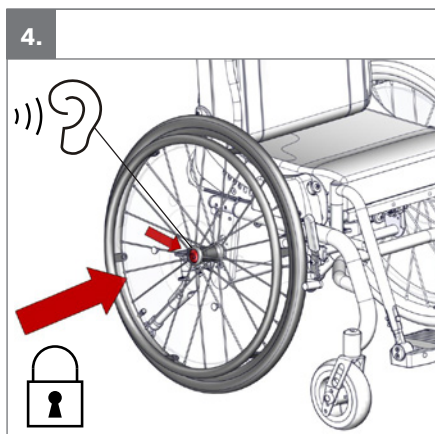
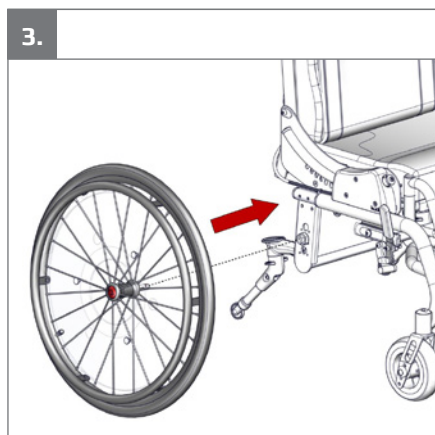
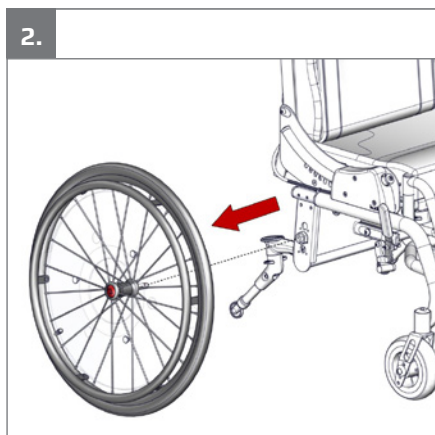
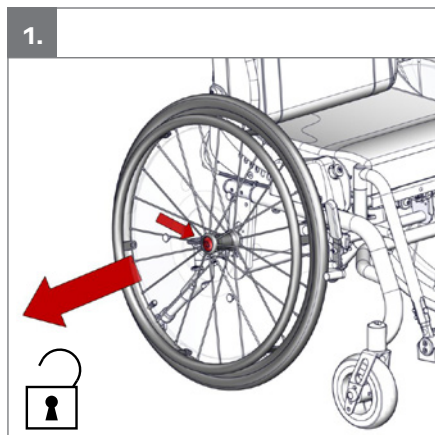
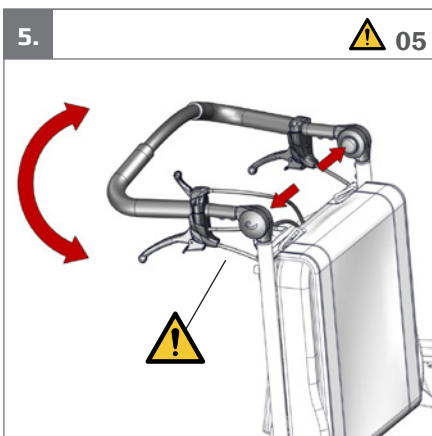
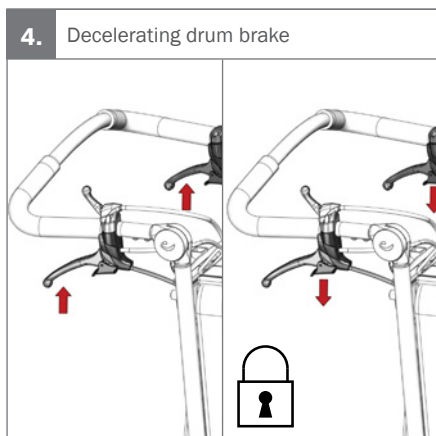
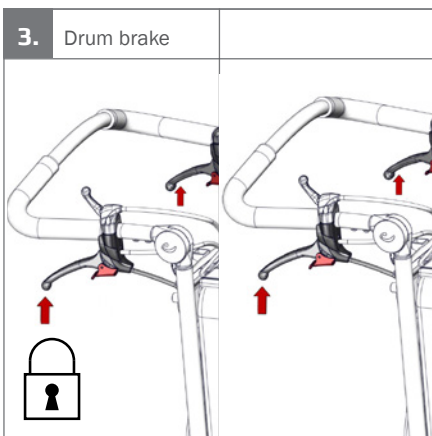
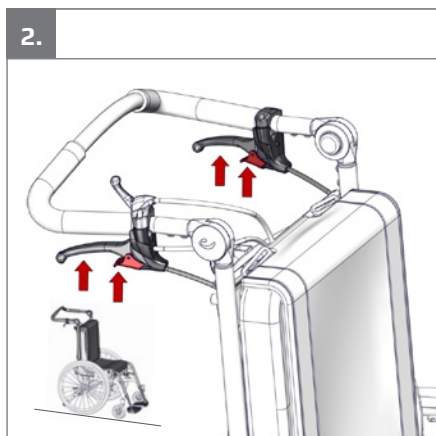
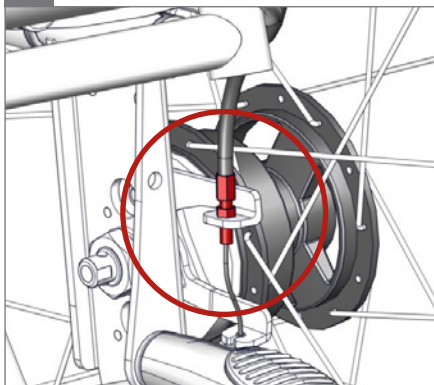
6.


7.

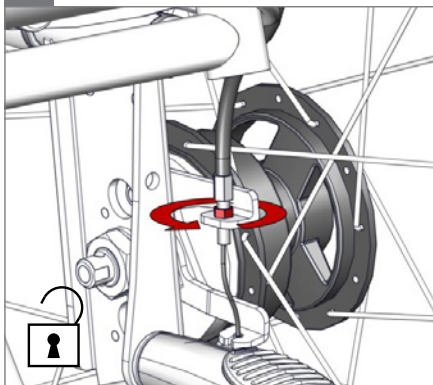
8 mm

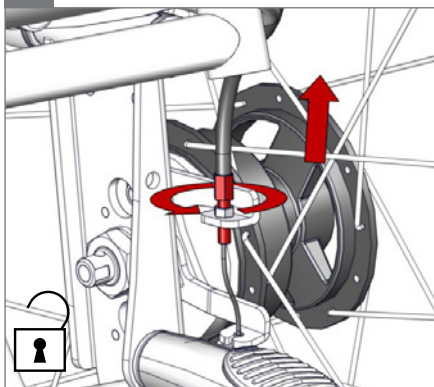8.

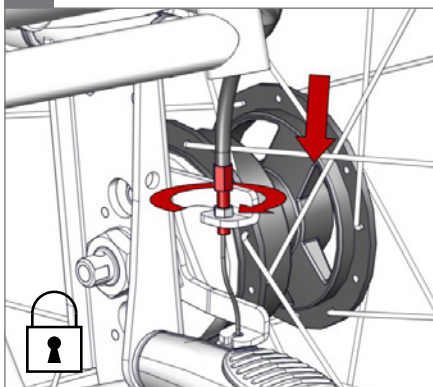
9.







10.




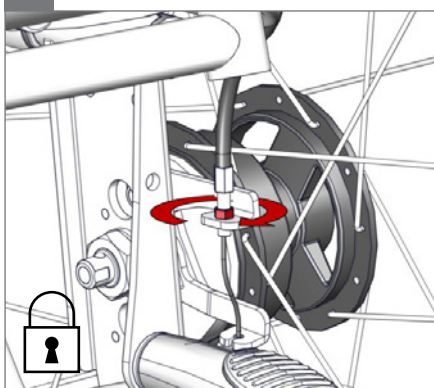
8 mm

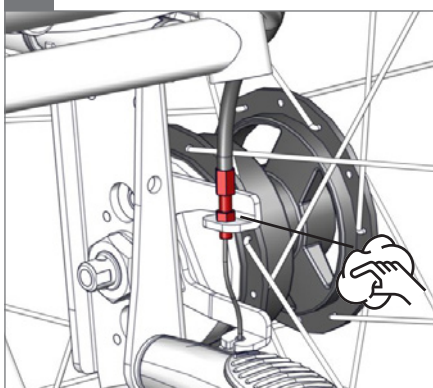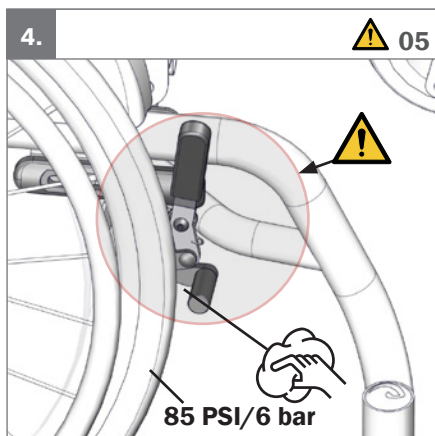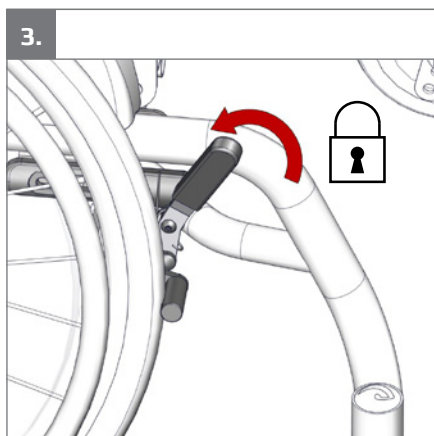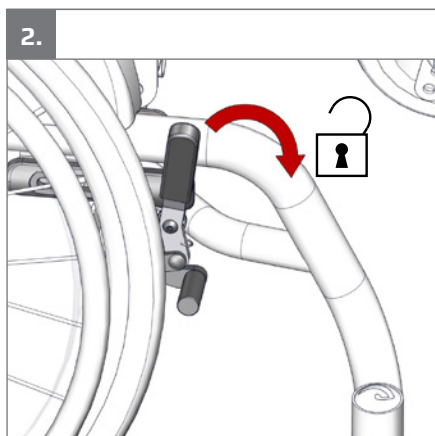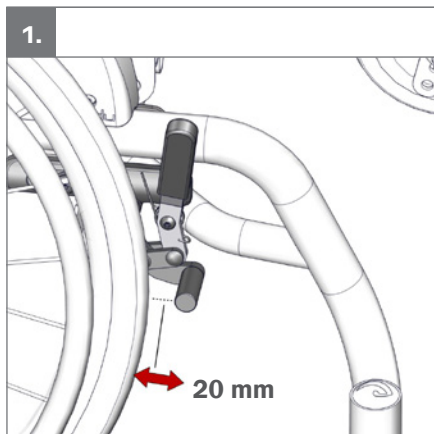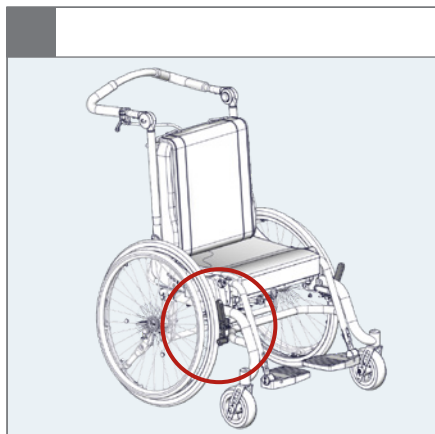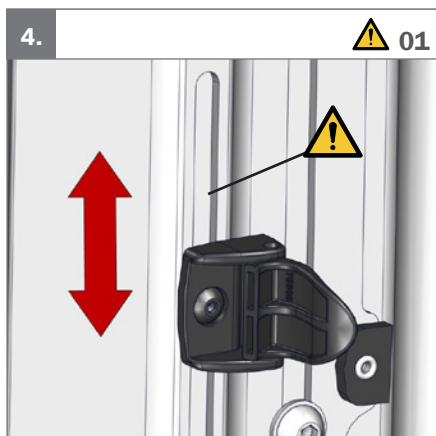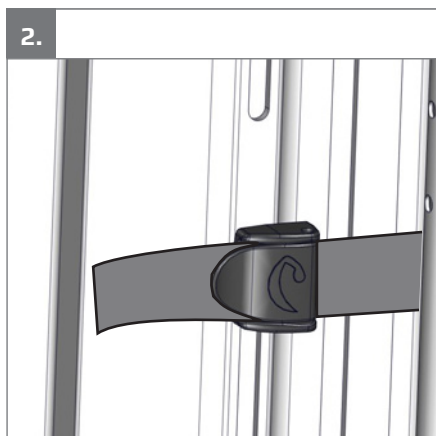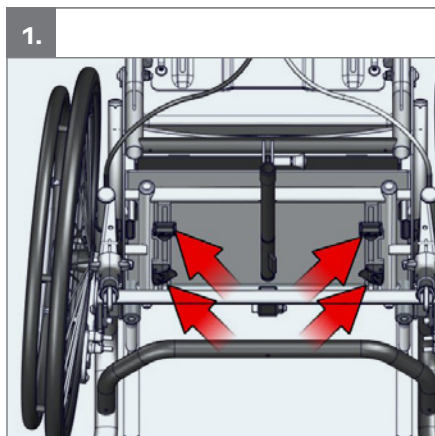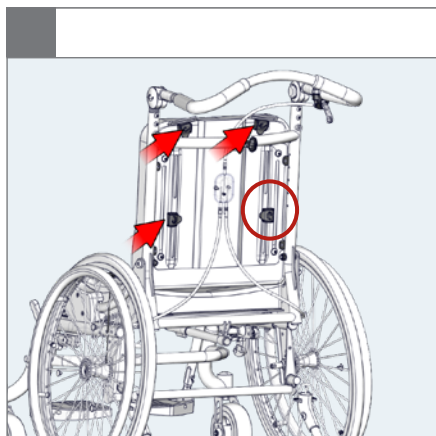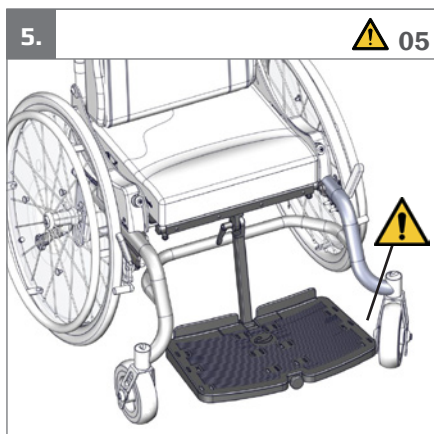
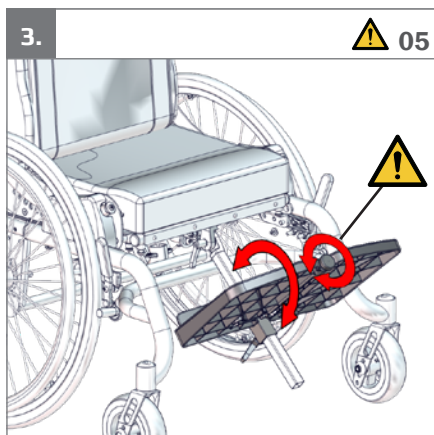
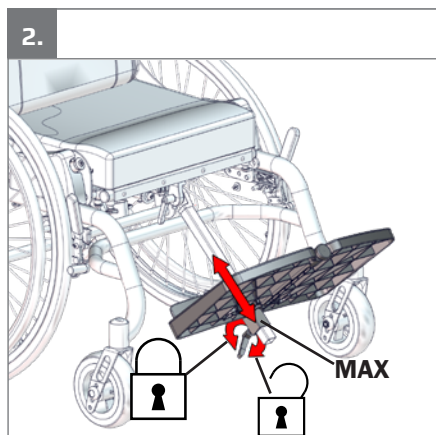
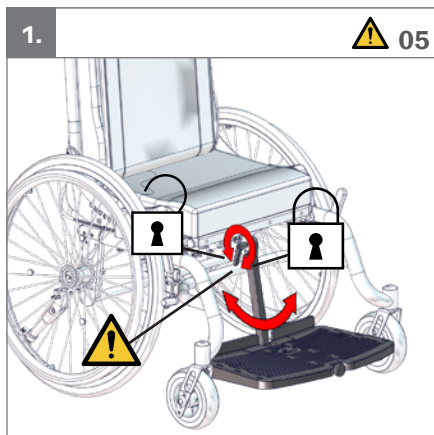
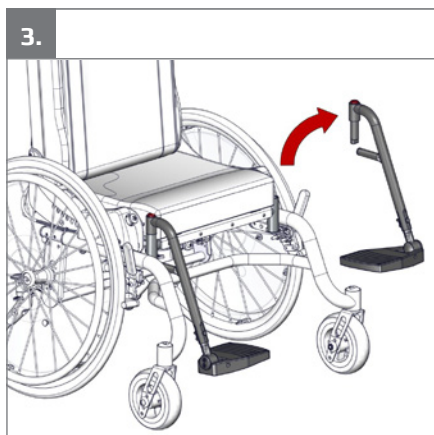
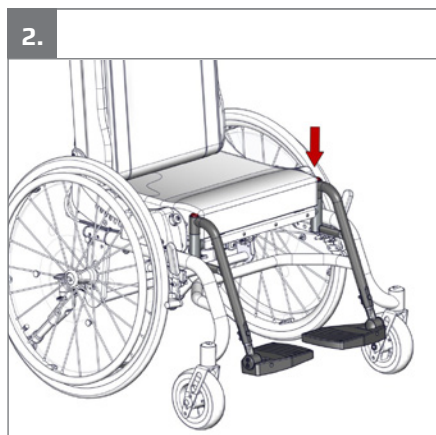
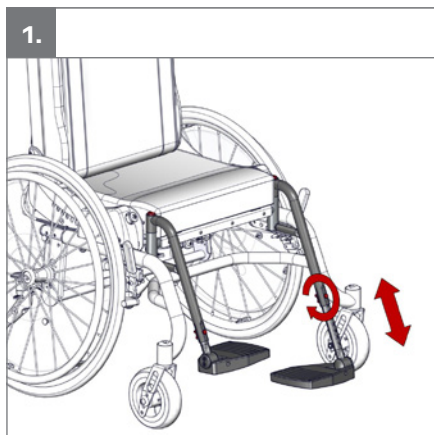
11.


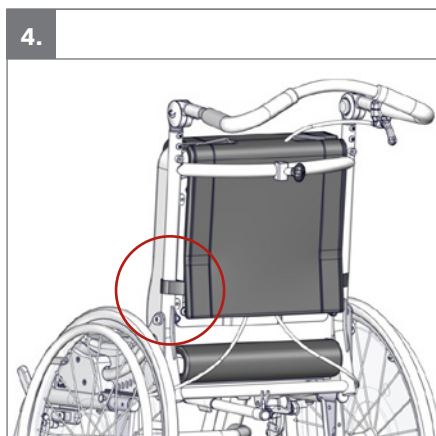
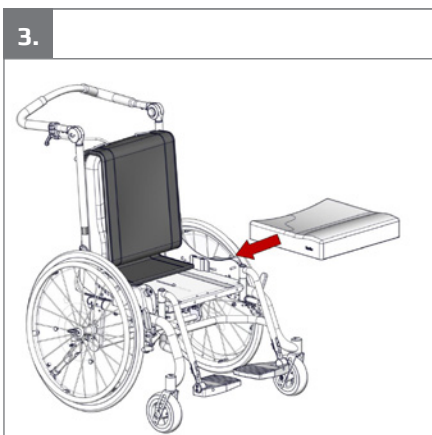
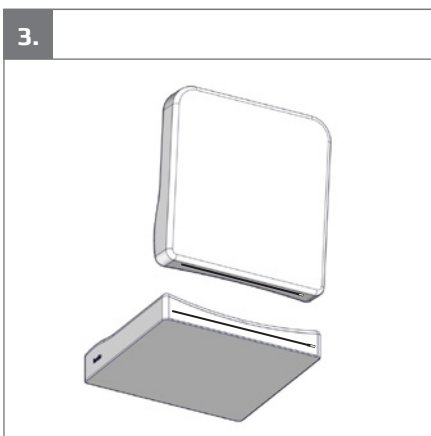
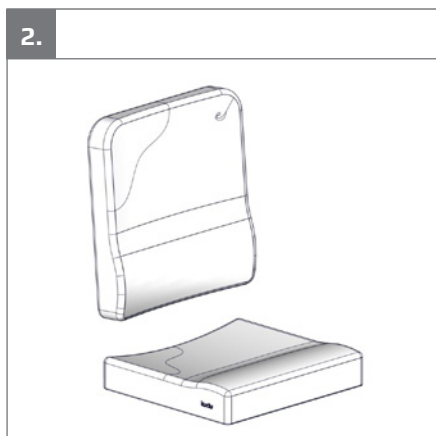
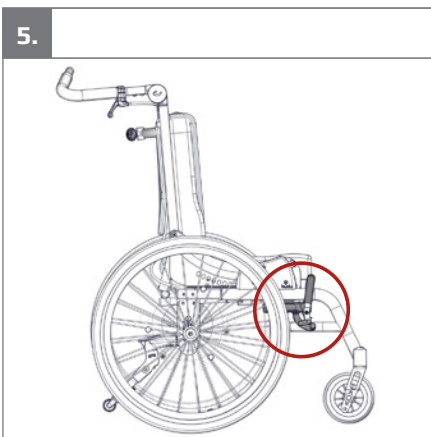
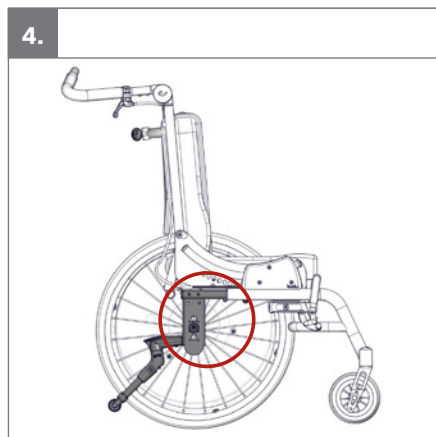
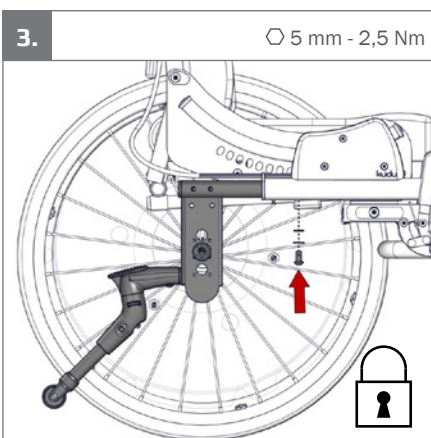
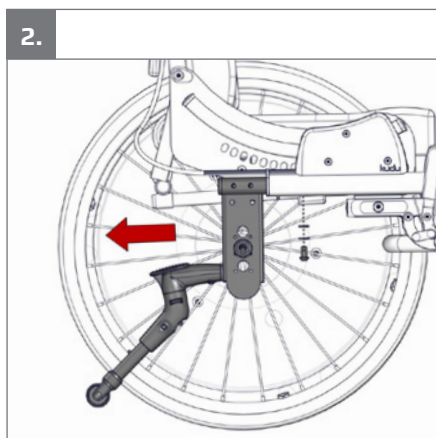
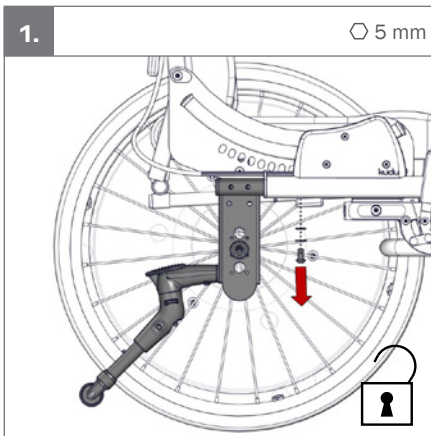
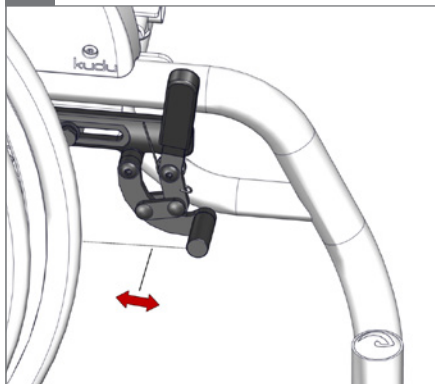
6.


7.

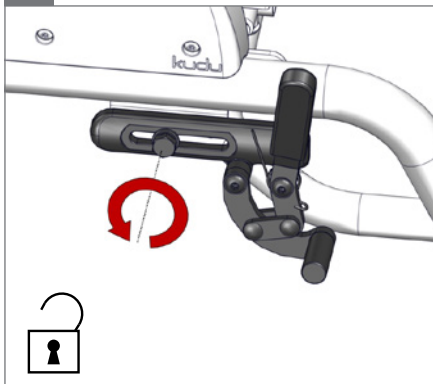13 mm


8.

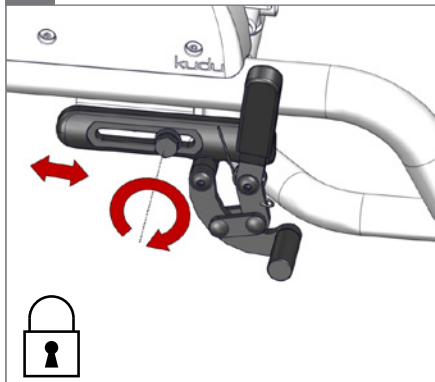13 mm - 2,5 Nm

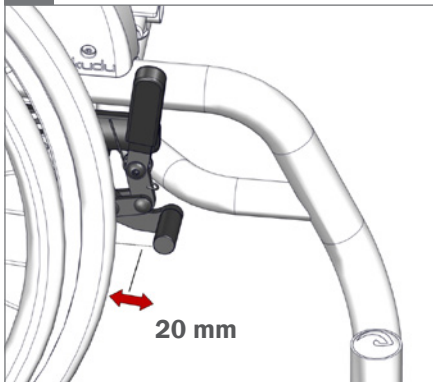9.

10.


	20"	22"	24"	
1	-	-	a 480	6"
2	-	a 480	c 505	
3	a 480	c 505	-	
2	-	-	b 505	7"
3	-	b 505	c 530	


	20"	22"	24"	6"	7"
Minimum psi	85	85	85	-	-
Maximum psi	145	145	145	-	36
Handrim diameter	445 mm (17½")	480 mm (19")	530 mm (21")	-	-

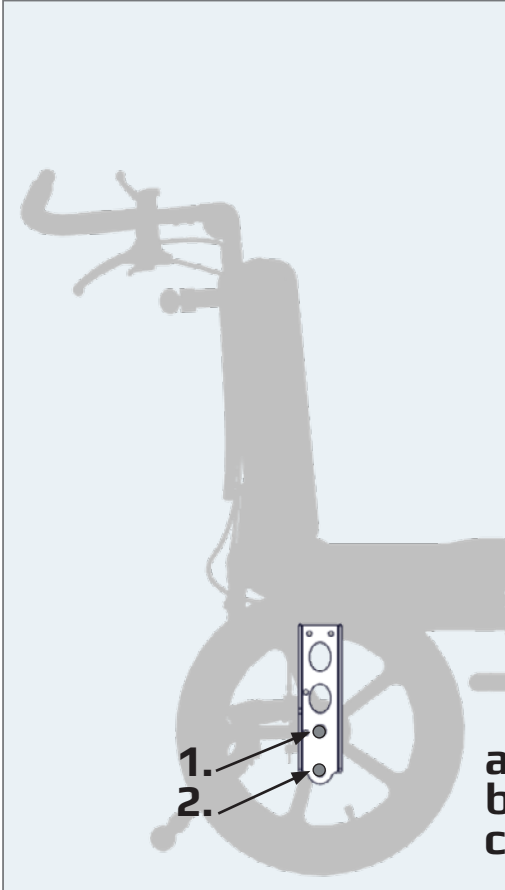
				
		12½"	16"	
1	-	a 480		6"
2	a 480	-		
1	-	-		7"
2	-	b 530		

	12½"	16"	6"
Minimum psi	-	35	-
Maximum psi	26	65	-

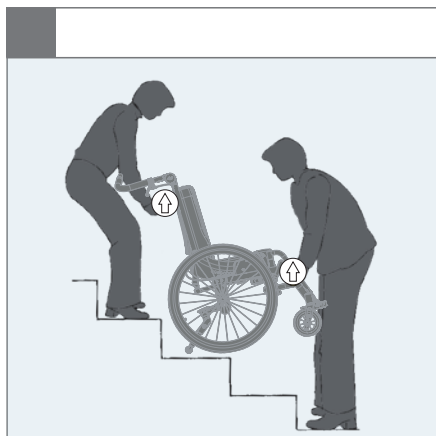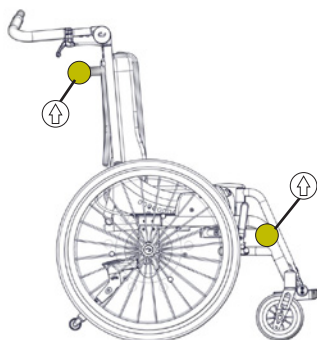
1.


2.


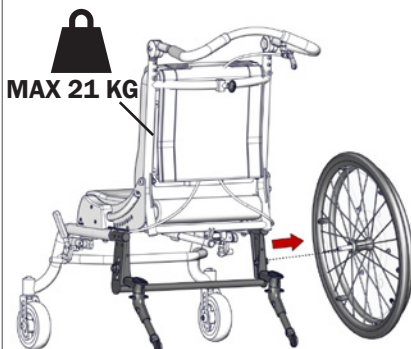
3.

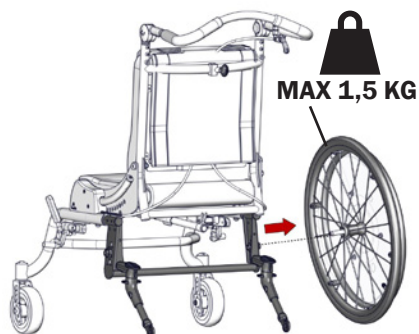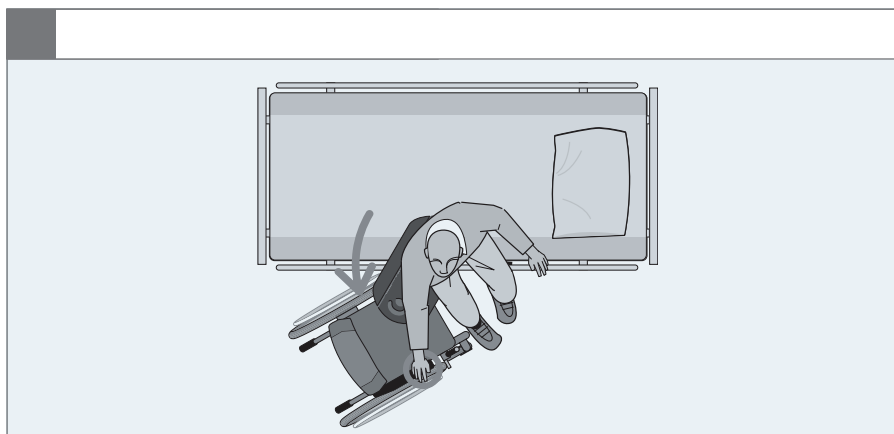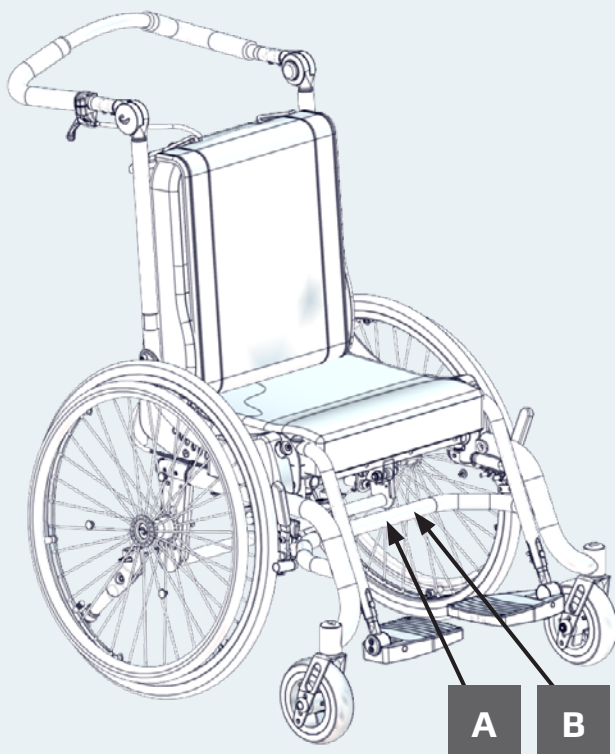4.

A

R82
by Etac

R82 A/S, Parallelvej 3
DK-8751 Gedved
etac.com
Made in xxxxxxx

MD

CE

UK
CA

YYYY-MM-DD

B

Posting group
Product name, size x

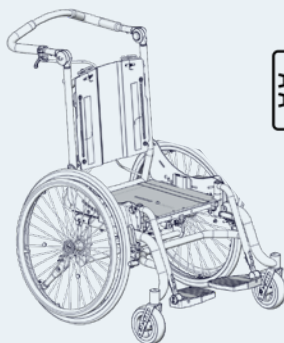
DMR ver.: xxxx

Max. user mass: xxx kg
Max. user mass: xxx kg
Max. rated load: xxx kg
Mass device: xxx kg

SR xxxxxxxxxxxxxxxx

REP xxxxxxxxxxxxxxxx

LDI (xx)xxxxxxxxxxxxxxxxxx(XX)XXXXXX(XX)XXXXXX

EN: Warning: This symbol appears in the User Guide along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger
01 = Entrapment Hazards
02 = Tilting Hazards
03 = Transportation in motor vehicles
05 = Squeezing Hazard

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
mm (inch) kg (lb)	 1	 2	 3	 4
 Mass device	19 kg (42 lb)	21 kg (46 lb)	23 kg (51 lb)	25 kg (55 lb)
 Maximum user mass	40 kg (88 lb)	50 kg (110 lb)	60 kg (132.3 lb)	70 kg (154.3 lb)
 Maximum user mass in transportation	40 kg (88 lb)	50 kg (110 lb)	60 kg (132.3 lb)	70 kg (154.3 lb)
 Maximum rated load	61 kg (134.4 lb)	73 kg (160.9 lb)	85 kg (187.3 lb)	97 kg (213.9 lb)
 Folded length	825 mm (32½")	825 mm (32½")	875 mm (34½")	875 mm (34½")
 Folded height	625 mm (24½")	625 mm (24½")	625 mm (24½")	625 mm (24½")
Mass of the heaviest part	13 kg (28.7 lb)	15 kg (33 lb)	17 kg (37.5 lb)	19 kg (41.9 lb)
 Leg to seat surface angle, centrally mounted footrest	55° – 180°	55° – 180°	55° – 180°	55° – 180°
 Leg to seat surface angle, parted footrest	105°	105°	105°	105°
 Armrest to seat distance	138–198 mm (5½–7¾")	138–198 mm (5½–7¾")	138–198 mm (5½–7¾")	138–198 mm (5½–7¾")
 Front location of armrest structure	590 mm (23")	590 mm (23")	640 mm (25")	640 mm (25")
 Horizontal location of axle	146–196 mm (5¾–7¾")	146–196 mm (5¾–7¾")	146–196 mm (5¾–7¾")	146–196 mm (5¾–7¾")
 Minimum turning radius	823 mm (32½")	823 mm (32½")	823 mm (32½")	823 mm (32½")
Required width of right angled corridor	1000 mm (39½")	1000 mm (39½")	1100 mm (43¼")	1100 mm (43¼")
 Stability sideways	10°	10°	10°	10°
 Static stability downhill	10°	10°	10°	10°
 Static stability uphill	10°	10°	10°	10°

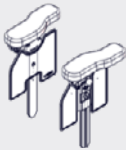
9996097001


9996097002

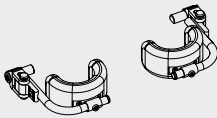
9996097003


9996097004

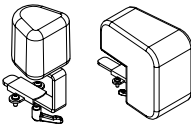
9996097005

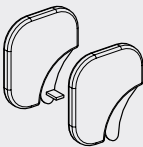
9996097006

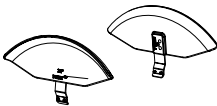9996097007

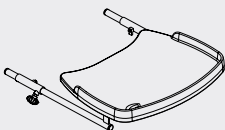
9996097008

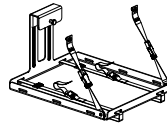
9996097009

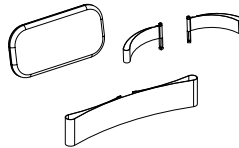9996097010

9996097011

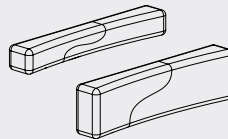9996097012

9996097013

9996097015

9996097740

9996097137

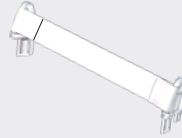
9996097187


9996098130

9996097748

9996098133

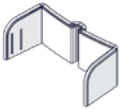
9996097338


9996098132

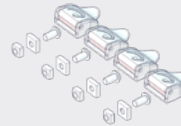
9996097339

9996098129

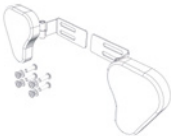
9996097741

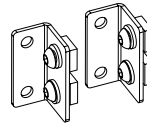9996098128

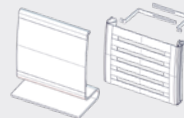
9996097016


9996097095

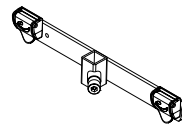
9996097608


9996098187

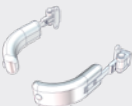
9996098131

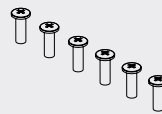
9996097167

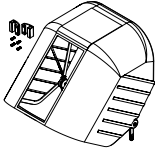
9996097170

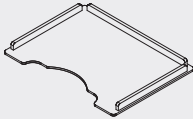
9996097157

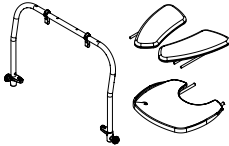
9996097706

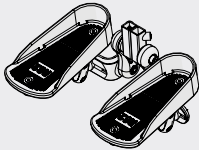
9996098274

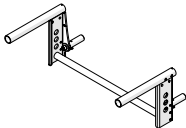9996098281

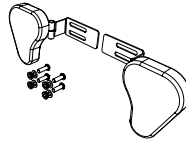9996097741

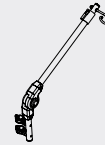9996098218

9996098252

9996098277

ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this user manual before use and save it for future reference.

Intended purpose

The device is a wheelchair for children and adults with disabilities.

Indications

The device can be used by children and adults with disabilities, who lack the ability to maintain a seated position or move around without an assistive device. Thus, the target group for the device is based on functional ability and not a specific health condition or age.

The user is the person sitting in the device. The caregiver is the person operating the device. Never leave the user unattended in this device. Ensure permanent supervision by a caregiver.

The use of the device will be discontinued more times during the day where the user will be lying down, standing or sitting in another assistive device or a conventional product.

For transportation the device is suitable for children and adults in the weight range of 22 kg to specified max user weight for transportation.

Contra indications

There are no known contra indications

Precautions

Special attention should be given to the considerations about the configuration of the product and the use of accessories for users with limitation in their:

- Sensory functions and the sensation of pain consider extra checks of the pressure caused by the support provided by the device and its accessories.
- Movement function (e.g. experience involuntary movements), consider using a device with a dynamic backrest.
- Ability to maintain a seated position, use the product with accessories (e.g. side supports, vests, belts or arm supports and a tray).
- Ability to maintain the position of their head, consider to use accessories or to use a product that provide support to the positioning of the head.

Application areas

The product is for indoor and outdoor use

Enclosed tools:

- 4+5 mm Allen key

- 13 mm spanner

Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745).

The CE mark must be removed when:

- Rebuilding the device
- Using the device outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings.

Combinations

When combining the device with other devices, the combination must not change the intended purpose of the devices or modify the devices in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the devices must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety.

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the user manual carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user manual.
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver
- Be aware of any MAX mark indications before adjustment
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Ensure that body parts are not trapped in openings for mounting of supports/ accessories.
- Information about transportation using strap tie downs or a Dahl Docking station with your R82 product can be found in the document "M1480 Transportation in Motor Vehicles" supplied with the product.
- Information about roadability can be found in the document "Roadability" supplied with the product.

Environment

- Observe the surface temperature on the product, before placing a user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to

measurements

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Removable parts without use of tool: wheels, foot support and cushions. The removal will have adverse effect on the device
- Activate the brakes before placing the user into the product
- Activate the anti-tips when the product is being used on inclines
- Activate the anti-tips before angling the seat and back
- Deactivate the anti-tips when forcing kerbs higher than 100 mm
- Ensure the stability of the product is maintained before placing the user in it
- We recommend you notify the user before angling the seat and back
- Ensure the wheels are fully-functional and secure before use
- The correct tyre pressure is printed on the tyre casing and listed in the section for tyre technical data.
- In case of a tyre puncture do not use the product until remedied
- Unoccupied device is suitable for land and air transport
- Attaching additional loads can adversely affect stability. Position of points intended to carry additional loads are specified in the relevant Mounting Guide referred to under section for accessories
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Inspect the product and all of its accessories and replace any worn out parts before use
- This device is tested for resistance to ignition at level non-ignition in accordance with applicable standards.
- The product might lose its flame resistant

characteristics if using other than original
R82: Cushions

- The product is approved for transportation in vehicles according to ISO 7176-19. More information can be found in the document "M1480 Transportation in Motor Vehicles" supplied with the product.
- In accordance with ISO 7176-19 annex D the product is evaluated regarding the overall rating of belt restraint accommodation.
Rating for: ease of properly positioning a three point belt restraint.
The product receives a score of "excellent".
Rating for: the degree to which proper three point belt restraint positioning can be achieved.
The product receives a score of "Good".

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the device. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this device in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Reuse

This device and its accessories/components are suitable for reuse.

Refurbishment for reuse

Before reuse or changing the user, follow the service checklist, maintenance and washing instructions and carry out regular inspections.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the device is inspected yearly, every time the device is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the device.

Service checklist:

Check and adjust the following as per their intended use:

- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- Seat tilt system is adjusted to run freely and without excess movement.
- Do not oil the tilt system.
- Wheels are moving freely and that all wheel locks/brakes are easily activated. Adjust if required.
- Quick release wheels are removable/replaceable.
- Anti-tips can be fixed in working and resting position.
- Tire pressure.
- Spoke guards is without cracks and secured to the wheel with the clips mounted on each guard.
- Before storage: Always deactivate wheel locks/brakes.
- Nuts and bolts on the product are tightened.
- Oil the swing-away parts.
- Product for cracks or signs for worn-out parts.
- All labels on the product are intact.

Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to:

Wipe off any residue and dirt from the device, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use.

Carry out a visual inspection, for any damaged or worn-out parts.

Washing

Main device

This device can be washed at 60°C using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the device. Unmount all accessories and wash them separately.

Tilt/rotate the device after washing to facilitate water drainage from within the device.

Fabric and covers

The cover can be washed at max. 60°C using a mild detergent in a washing machine.

Separate foam and cover before wash.

Foam

The foam inserts must only be washed by hand. Use warm water and a mild detergent/soap without chlorine and let all parts dry before use.

During and after washing the foam, do not bend, squeeze or wring it, as it may negatively affect the shape.

Handwash

This device and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The device can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the device, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Materials

- Steel
- Aluminium
- Plastic
- Rubber

- Wood
- Fabric
- Foam

Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

Safety product

Due to variant and settings the wheelchair may become larger than the recommended dimensions. Compliance with recommended overall width and overall length ensures unhindered use of emergency escape routes. (See table 1)

Table 1

Recommended dimensions	Ready for use (mm)
Overall length – maximum	1200
Overall width – maximum	700
Overall height – maximum	1200
Turning diameter – maximum	1000
Pivot width – maximum	1300
Ground clearance - minimum	30

Product safety

Due to variant and settings the wheelchair may not meet the limits for wheelchairs transportable by train. Measures can be taken to verify whether the specific product in question meets the limits. (See table 2)

Table 2

Wheelchairs transportable by train - limits	
Overall length – maximum	1200 mm + 50 mm for feet
Overall width – maximum	700 mm + 50 mm on each side for hands
Overall height – maximum	1375 mm (including 95th percentile male occupant)
Turning diameter – maximum	1500 mm
Smallest wheels	Accommodate a gap of 75 mm horizontal and 50 mm vertical
Weight – maximum	200 kg for manual wheelchair and occupant (including baggage)
Obstacle height that can be overcome – maximum	50 mm
Ground clearance - minimum	60 mm under foot rest with upward slope angle of 10° on top for going forward
Safe slope on which the wheelchair remain stable - maximum	6° dynamic stability in all directions 9° static stability in all directions, including with brake applied

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**Etac Ltd.**
**Gelbart AG**