
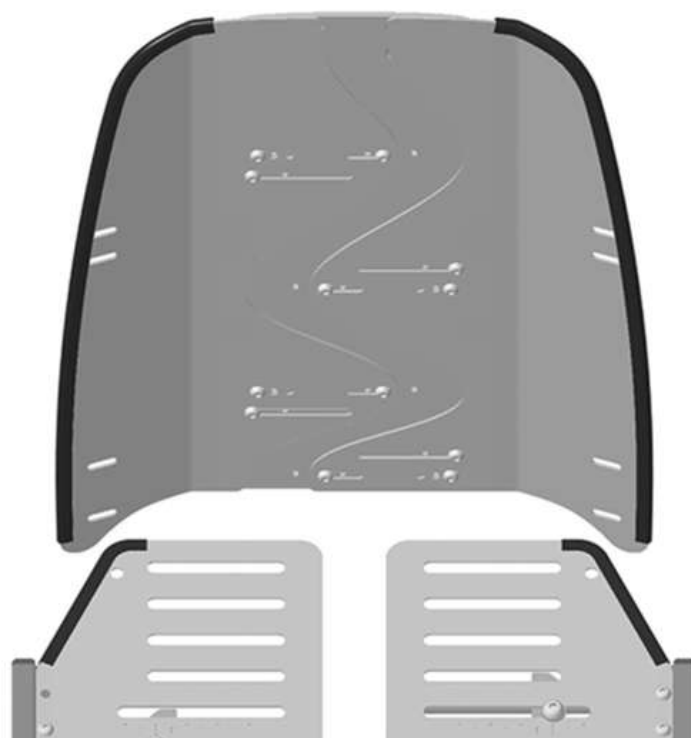
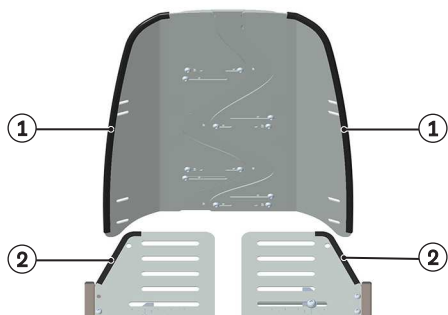
491S75=ST435

DE	Gebruuchsanweisung	5
EN	Instructions for use	15
FR	Instructions d'utilisation	25
IT	Istruzioni per l'uso	35
ES	Instrucciones de uso	45
PT	Instruções de utilização	55
NL	Gebruiksaanwijzing	65
SV	Bruksanvisning	75
DA	Brugsanvisning	85
NO	Bruksanvisning	95
FI	Käyttöohje	105
PL	Instrukcja użytkowania	115
HU	Használati útmutató	125
CS	Návod k použití	135
HR	Upute za uporabu	145
SL	Navodila za uporabo	155
SK	Návod na použitie	165
TR	Kullanım kılavuzu	175
RU	Руководство по применению	185
ZH	使用说明书	197
ZH ^{HK}	使用說明書	205
KO	사용 설명서	215


1

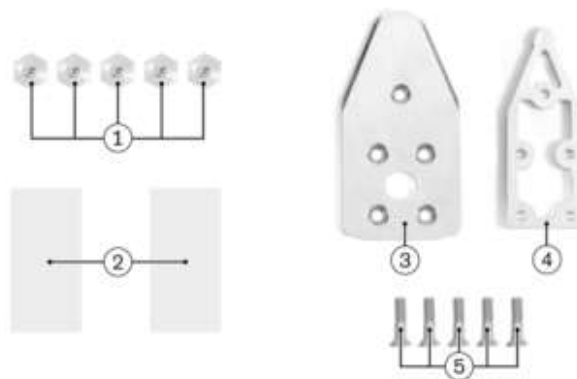
2


3

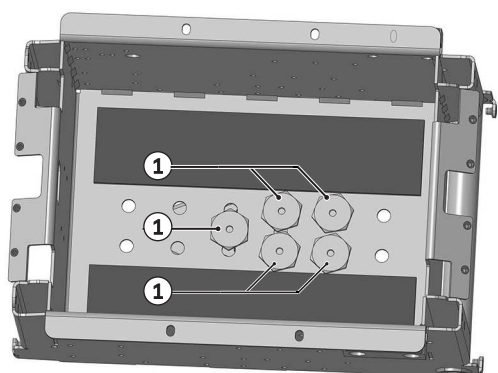4

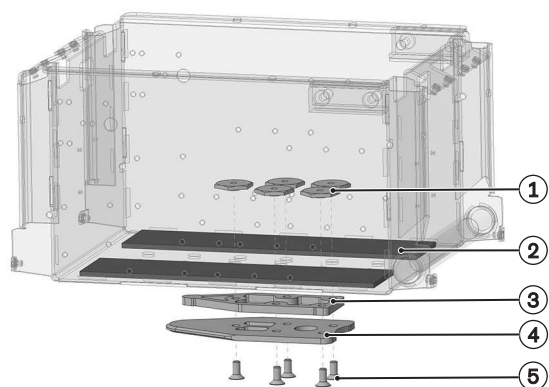5


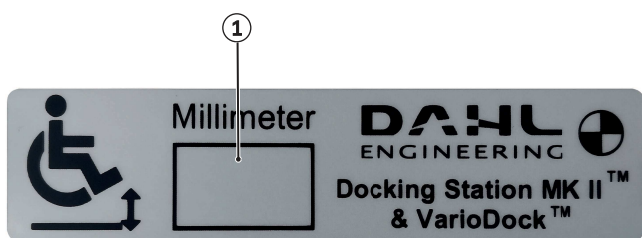
7

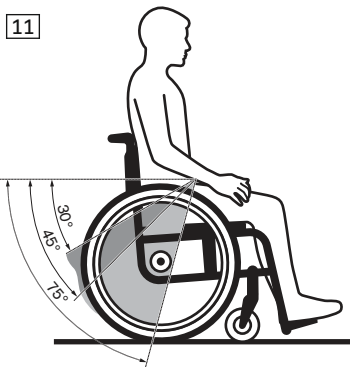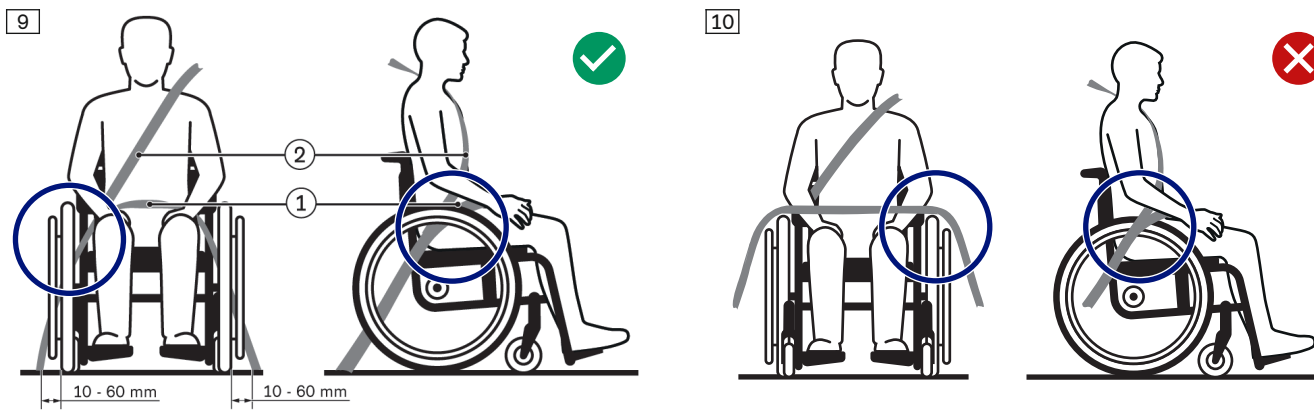8

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DE

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Table of contents

EN

1	General	16
1.1	Further Applicable Documents	16
1.2	Target group	16
2	Safety-related information	17
2.1	Meaning of warning levels	17
2.2	Safety instructions for qualified personnel	17
2.3	Safety instructions for use	17
3	Product description	18
4	Intended use	18
4.1	Indications for use	18
4.2	Environmental conditions	18
4.3	Product combinations	18
5	Scope of delivery	18
6	Accessories	19
7	Preparing the product for use	19
7.1	Required tools	19
7.2	Installation	20
8	Use	21
8.1	Prescribed use of head supports	21
8.2	Use in vehicles	21
9	Cleaning	23
10	Maintenance	23
11	Repair	23
12	Disposal	24
13	Legal information	24
13.1	Liability	24
13.2	Warranty	24
13.3	Local Legal Information	24

1 General

INFORMATION

Date of last update: 2025-05-12

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

INFORMATION

- ▶ New information regarding product safety and product recalls can be obtained at ccc@ottobock.com or from the manufacturer's service department (see www.ottobock.com and www.dahlengineering.dk for addresses).
- ▶ You can request this document as a PDF file at ccc@ottobock.com or from the manufacturer's service department. The PDF file can also be displayed in a larger size.

These instructions for use provide you with important information on the installation of attachments (edge protection), which are required for the use of the "Dahl Docking System" option on approved Ottobock power wheelchairs.

They also describe how the locking plate offered by Dahl Engineering ApS is installed on approved Ottobock power wheelchairs as part of the "Dahl Docking System" option.

As qualified personnel, please note the following:

- Ottobock only provides the attachments required for the use of the Dahl Docking System. These are part of the Ottobock mounting kit for the "Dahl Docking System" option. Information on this is provided in the "Scope of delivery" section (see page 18).
- The "Accessories" section (see page 19) tells you which parts are required from Dahl Engineering ApS and how these parts can be obtained.
- Please read these instructions for use prior to installing the Ottobock mounting kit for the "Dahl Docking System" option.
- Hand over all instructions for use to the user. The user must have read and understood these instructions for use and the instructions for use from the manufacturer Dahl Engineering ApS before using the "Dahl Docking System" option. The user should store the documents in the vehicle in an easily accessible location.

1.1 Further Applicable Documents

The following service, installation and operating instructions provided by Ottobock and Dahl Engineering ApS must be observed in their entirety depending on the product combination:

Item	Document number	Designation	Designation in this document ¹⁾
1	647H2021	Instructions for use (for wheelchair 490E75=2_*)	Instructions for use for the wheelchair
2	647G2021	Service manual (for the wheelchair 490E75=2_*)	Wheelchair service manual
3	503747	DAHL DOCKING STATION MK II SERVICE PLAN installation and operating instructions	Instructions for use for the docking system
4	503752	DAHL VARIODOCK™ service plan installation and operating instructions	Instructions for use for the docking system

¹⁾: To simplify the text, the other applicable documents specified in the table are referenced in the rest of these instructions for use according to the designations in the "Designation in this document" column at the relevant points in the text.

1.2 Target group

This document is intended for authorised qualified personnel who install and adjust the "Dahl Docking System" option on the power wheelchair and for the power wheelchair user.

All installation and adjustment tasks in the “Preparing the product for use” section (see page 19) may only be carried out by authorised, qualified personnel. All manufacturer specifications and all applicable legal provisions must be complied with. Please contact the manufacturer’s service department for further information.

2 Safety-related information

2.1 Meaning of warning levels

WARNING! Failure to follow instructions labelled in this way can lead to serious accidents and injuries.

CAUTION! Failure to follow instructions labelled in this way can lead to accidents and injuries.

NOTICE! Failure to follow instructions labelled in this way can result in technical damage.

2.2 Safety instructions for qualified personnel

CAUTION!

Risk of injury due to moving parts

- ▶ Please note the design-related pinching and shearing points between moving parts of the product, e.g. in the area between the seat bottom and the mobility base. Keep all body parts away from danger areas during adjustments so that they cannot be pushed, pinched or crushed. Perform the work with assistance from a helper if necessary.

CAUTION!

Risk of injury due to non-ergonomic work

- ▶ Some parts of the product, such as the batteries, frame, seat and motors, are very heavy. Ensure ergonomically correct lifting of these parts. Use sufficiently large hoisting devices or perform this work with the assistance of a helper.
- ▶ If it is necessary to work underneath raised parts or equipment, make sure they are secured by suitable means so that they cannot come loose, shift or fall down.
- ▶ When using lifting platforms, ensure that the power wheelchair is centred on the platform and that no parts protrude into the danger zone.

CAUTION!

Risk of injury due to uncontrolled movements

- ▶ Switch off the control device of the product and deactivate the main fuse for all adjustment work. Functional tests of the electrical components are excepted from this rule.

NOTICE!

Danger of product damage due to improper loading

- ▶ Ensure that the padding and textile components on the product are adequately protected against mechanical, chemical and thermal damage during all work.

2.3 Safety instructions for use

WARNING!

Risk of injury due to improper handling of the product

Improper handling of the product can lead to serious injuries and damage to the product. Damage and wear and tear to the product can impair its safety, functionality and reliability and lead to additional hazards.

- ▶ The product may only be used by a user or attendant who has been instructed accordingly.
- ▶ These instructions for use apply only in conjunction with the instructions for use for the wheelchair and the instructions for use for the docking system. Read all information regarding use carefully before using the product and observe all safety notes.
- ▶ Check that the product is working correctly before each use.
- ▶ The product may not be used if defects are noted. This applies in particular in case of instability of the product or changed driving characteristics as well as problems with the user’s sitting position or the stability of the seat. Inform the qualified personnel promptly to rectify defects.

3 Product description

Information on the function of the attachments supplied by Ottobock (edge protection)

The attachments supplied by Ottobock must be fitted on the wheelchair prior to the use of the Dahl Docking System, otherwise the belts of the occupant restraint system could become damaged and their function could be adversely affected. The attachments supplied reduce the mechanical forces which can act on the belt system of the occupant restraint system.

Information on the function of the “Dahl Docking System” option

With the help of the Dahl Docking System, wheelchairs can be attached to the floor of appropriately equipped vehicles so they can be used as a seat in the vehicle. For this purpose, the locking plate supplied by Dahl Engineering ApS must first be installed under the wheelchair. Further information can be found in the “Installation” section (see page 20).

One of the following docking stations must be installed in the vehicle by a suitably qualified vehicle retrofitter (see instructions for use for the docking system).

When the wheelchair is driven into the docking station, the wheelchair is secured on the floor of the vehicle using the locking plate in the docking station. For more detailed information on the overall function of the Dahl Docking System, see the instructions for use for the docking system.

Dahl Engineering ApS offers two docking systems (see page 20 “Accessories” section):

- Dahl Docking MK II
- Dahl VarioDock™ (height-adjustable)

The locking plates and wheelchair adaptation kits are identical for both docking stations and can therefore be used for securing a wheelchair with both systems.

For the “VarioDock™” system, check the compatibility with the wheelchair ground clearance in advance, as it is 54 mm wider and 1 mm higher than the “Docking MK II” system.

4 Intended use

4.1 Indications for use

The wheelchair specified in the “Product combinations” section (see page 18) was tested in combination with a “Dahl Docking System”, the “accessories” for the docking system (see page 19 “Accessories” section) and the attachments (see page 18 “Scope of delivery” section) according to the ISO 7176-19 standard and therefore meets all crash safety requirements for use as a seat in a vehicle. The wheelchair has been tested only in the forward direction of travel and is therefore approved exclusively for use as a **forward** vehicle seat.

4.2 Environmental conditions

The Ottobock mounting kit is designed for the same environmental conditions as the wheelchair on which it is installed.

- Observe the corresponding information in the instructions for use for the wheelchair.

4.3 Product combinations

The “Dahl Docking System” option is approved for the following Ottobock products:

- Type: Juvo B7
- Reference number: 490E75=2_*

5 Scope of delivery

The scope of delivery of the Ottobock “Dahl Docking System” mounting kit, order number 491S75=ST435, includes (see fig. 1):

Item	Article designation	Quantity	Order number
1	Edge protection for APS back plate	2	491S75=ST435
2	Edge protection for APS seat plate	2	
Not illustrated	Instructions for use	1	

6 Accessories

Adaption set supplied by Dahl Engineering ApS for the wheelchair 490E75=2_*/Ottobock (see fig. 2):

Item	Article designation	Quantity	Order number
1	M8 hexagon nut	5	503261**
2	Spacer strip, PVC, 5 mm	2	

Parts from the basic kit supplied by Dahl Engineering ApS (see fig. 2)

Item	Article designation	Quantity	Order number
3	Locking plate	1	500561**
4	Intermediate piece, 8 mm	1	500673**
5	Mounting screws M8x45, Torx	5	502800**

Overview of the compatible docking stations (see fig. 7, see fig. 8):

Fig.	Article designation	Quantity	Order number
7	Docking Station Mk.II complete	1	501750**
8	Dahl VarioDock	1	503600**

Overview of the required installation and operating instructions for the above-mentioned docking stations (not illustrated):

Fig.	Article designation	Quantity	Order number
Not illustrated	DAHL DOCKING STATION MK II SERVICE PLAN installation and operating instructions	1	503747**
Not illustrated	DAHL VARIODOCK™ service plan installation and operating instructions	1	503752**

** Order numbers for Dahl Engineering ApS. Order via: Dahl Engineering ApS | Løvevej 3 | DK-7700 Thisted | Denmark | Tel: +45 96 18 00 77 | sales@dahlengineering.dk | <https://dahlengineering.dk>

7 Preparing the product for use

WARNING

Incorrect installation and adjustment work

User injuries due to defective safety functions

- All installation and adjustment tasks described in the "Preparing the product for use" section may only be carried out by qualified personnel.
- Only the installation tasks and adjustments described in these instructions for use may be carried out on the wheelchair. If you have any questions, contact the manufacturer's service department.

7.1 Required tools

The following tools are required for adjustments and maintenance work:

INFORMATION

The following list only includes tools which are required for the direct installation of the parts specified in the scope of delivery of Dahl Engineering ApS and Ottobock. For information on tools for preparatory work (e.g. to remove the batteries), please see the wheelchair service manual.

- Hexagon key, size 6
- Ring and open-ended spanners, size 8, 24
- Torque wrench (measurement ranges 5 – 50 Nm)
- Torx screwdriver, size T45
- Metal saw
- Scissors or cutter knife
- Liquid thread lock, "medium strength" (e.g. Loctite 241)

7.2 Installation

Safety instructions for the installation of the Dahl parts on the Ottobock power wheelchair

WARNING

Failure to observe the information provided by Dahl Engineering ApS and Ottobock

Severe injury in case of accidents due to installation errors

- ▶ During all installation work, in addition to the safety notes contained in these instructions for use, the safety notes in the instructions for use for the docking system must be observed.
- ▶ Changes that go beyond the steps described in the instructions for use provided by Ottobock and Dahl Engineering ApS may not be carried out without consulting the respective manufacturers! This applies to any changes to the Dahl Docking System, the anchoring devices in the vehicle and also to the wheelchair.

CAUTION

Incorrect use of mounting screws

Severe injuries due to breakage of the Torx mounting screws

- ▶ Only use the Torx mounting screws supplied by Dahl Engineering ApS. Other standard screws are not strong enough to hold the locking plate in case of a collision.
- ▶ Please note that the Torx mounting screws have to be shortened during installation. Do not damage the thread when shortening the screws.
- ▶ Please note that the Torx mounting screws have to be secured with “medium strength” liquid thread lock (e.g. Loctite 241) during installation.

INFORMATION

Please note that the following steps refer in part to instructions in the wheelchair service manual. You can request the PDF file of the wheelchair service manual from the Customer Care Center (CCC) at ccc@ottobock.com or from the manufacturer's service department. Before starting installation work, ensure that you have the wheelchair service manual to hand:

- ▶ Wheelchair service manual for 490E75=2_*: 647G2021

Installing the locking plate from Dahl Engineering ApS on the drive support

- 1) **Only in the equipment version with power seat functions:** Move the seat to the lowest seat height.
- 2) Turn the control device off and deactivate the fuse.
- 3) Slightly jack up the power wheelchair (see wheelchair service manual).
- 4) Prepare access to the drive support. To do this, fold the seat up and back (see wheelchair service manual).
- 5) Remove the battery cover and batteries (see wheelchair service manual).
→ The floor of the drive support is now freely accessible.
- 6) Determine the correct installation position for the locking plate (see fig. 2, item 3; see fig. 3).
- 7) Have the 5 hexagon nuts (see fig. 2, item 1) ready for subsequent installation (see fig. 4, item 1).
- 8) Place the spacer strips on the floor of the drive support in accordance with the illustration (see fig. 4, item 2).
- 9) Have the intermediate piece (see fig. 2, item 4; see fig. 4, item 3) and locking plate (see fig. 2, item 3; see fig. 4, item 4) ready. Also have the 5 Torx mounting screws ready for this purpose (see fig. 4, item 5).
- 10) Insert the Torx mounting screws one after the other from below through the locking plate, the intermediate piece and the 5 holes of the drive support (see fig. 4).
- 11) Provisionally screw the Torx mounting screws onto the drive support from above, using the 5 hexagon nuts.
- 12) **If needed:** Mark the excess. Remove the screws, shorten them and carefully deburr the top.
WARNING! The screws must be flush with the hexagon nuts. They may neither protrude beyond the hexagon nuts nor be too short to achieve their full holding force.
- 13) Reinsert the Torx mounting screws from below and secure them with “medium strength” liquid thread lock (e.g. Loctite 241).
- 14) Screw the 5 hexagon nuts onto the Torx mounting screws from above and tighten to **18 Nm**.
- 15) Reinsert the batteries (see wheelchair service manual).
- 16) Close the battery cover again (see wheelchair service manual).
- 17) Reactivate the fuse.
→ The locking plate of the Dahl Docking System is installed under the power wheelchair.

Adjustment and verification of functionality (see fig. 6)

Verification of functionality is required before using the locking plate in the Dahl docking station. Safe locking of the Dahl docking station has to be checked. For more information, see the instructions for use for the docking system.

Installing attachments supplied by Ottobock (edge protection) on the APS

- 1) Cut the edge protection to size if required. The length is determined by the length of the edge of the seat plate and the back plate.
- 2) Position the edge protection on the edges of the back plate (see fig. 1, item 1) and the seat plate (see fig. 1, item 2) and press it on by hand.

Applying the label

- 1) Once the wheelchair is equipped with the locking plate for the Dahl Docking System, affix the label to the back support of the wheelchair so it is readily visible.
- 2) After installation, measure the ground clearance with the correct tyre pressure and note this value on the label (see fig. 5, item 1).

INFORMATION

- The value stated on the label indicates the ground clearance of the wheelchair (-1 to +2 mm). In the case of the electrically height-adjustable VarioDock docking system, this is the height that must be set as the height value on the display of the control unit. For the MK II, this is the final height at which the docking unit must be installed in order to properly connect the wheelchair to the docking station.

8 Use

8.1 Prescribed use of head supports

⚠ WARNING

Failure to follow Ottobock instructions

Serious injuries in case of accidents due to user error

- When using the Dahl Docking System with approved Ottobock power wheelchairs, the installation of an Ottobock head support approved for use in combination with the Dahl Docking System is mandatory.
- Note the list of possible head supports below.

The following head supports are approved for use in connection with the Dahl Docking System:

Code number	Description	Article number
EL033	Comfort Plus, size S	476L52=SK105
EL034	Comfort Plus, size M	476L52=SK110
EL035	Comfort Plus, size L	476L52=SK115
EL036	Combo (3-pad), size S	476L52=SK150
EL037	Combo (3-pad), size L	476L52=SK155

8.2 Use in vehicles

⚠ WARNING

Failure to observe the information provided by Dahl Engineering ApS

Serious injuries in case of accidents due to user error

- Before using the Dahl Docking System in vehicles, read the instructions for use for the docking system and in particular the safety notes of the manufacturer.
- Only use the Dahl Docking System if the attachments supplied by Ottobock are installed on the power wheelchair. Otherwise the belts of the occupant restraint system could become damaged and their function could be adversely affected.

⚠ WARNING**Incorrect positioning in vehicles**

Serious injuries in case of accidents due to user error

- ▶ Ensure that the vehicle is suitable for transporting a person in a wheelchair. Please note that the load capacity of the vehicle must take into account the mass of the wheelchair.
- ▶ Make sure that the vehicle has the necessary equipment and that it is compatible with your wheelchair.
- ▶ Make sure that the options for getting into and out of the vehicle are suitable for your wheelchair type.
- ▶ Positioning of the product in the vehicle may only be performed by the person driving the vehicle.
- ▶ The product must always face forwards when it is used as a seat in a vehicle.
- ▶ Please also observe the positioning information in the instructions for use supplied with the docking system from the manufacturer Dahl Engineering ApS.

⚠ WARNING**Use of the positioning belt system or positioning aids as an occupant restraint system in a vehicle is forbidden**

Serious injuries due to improper handling of the product

- ▶ Under no circumstances may the positioning belts and positioning aids that come with the product be used as part of an occupant restraint system in a vehicle.
- ▶ Note that the positioning belts and positioning aids that come with the product are intended only as additional support for the user sitting in the product.

⚠ WARNING**Using the occupant restraint system in the vehicle in combination with the docking system**

Serious injuries due to improper handling of the product

- ▶ When using the docking system, only use the 3-point restraint systems installed in the vehicle for occupant restraint.
- ▶ In addition to the information and safety notes in these instructions for use, observe the safety notes in the instructions for use for the docking system.

⚠ WARNING**Using the product with installed options**

Severe injury in case of accidents due to options coming loose

- ▶ Before using the product as a seat in a vehicle, remove options that need to be taken off for safe transportation in the vehicle. Observe the information in the "Use in vehicles for transporting persons with reduced mobility" section in the instructions for use for the wheelchair.
- ▶ Stow all dismantled options securely in the vehicle.

When using the power wheelchair as a seat in a vehicle, attention must also be paid to the information provided in the instructions for use for the wheelchair, in particular the "Use in vehicles for transporting persons with reduced mobility" section.

Preparing the wheelchair

- 1) Attach a head support that is approved for use to the wheelchair if no other head support is permanently installed on the wheelchair.
- 2) Before each use, remove accessories that are not approved for transport in a vehicle or that may come loose while driving.
- 3) Please note that accessories and components such as a cane holder, cup holder, storage bag, storage basket or luggage carrier must not be used during transportation and must remain empty. Stow objects that have been removed securely in the vehicle.

Reference to Dahl Engineering ApS documents

When using the Dahl Docking System in a vehicle, observe the operating instructions in the instructions for use for the docking system from the manufacturer Dahl Engineering ApS, in addition to the following information. This applies in particular to content describing the function and operation of the respective docking system, such as:

- Positioning and attachment of the wheelchair in the docking station
- Securing the wheelchair in the docking station

- Releasing the wheelchair from the docking station
- General operation and adjustment of the docking system

Positioning the wheelchair

- 1) Position the wheelchair facing forwards in the vehicle and drive into the Dahl docking station.
- 2) Activate the locking device (see instructions for use for the docking system).
- 3) Check that the locking device is working properly and is secure.
- 4) Move any seat settings such as seat height and seat inclination (tilt) all the way down or to the horizontal position.
- 5) Move the back support to an upright position.
- 6) Always activate the wheel locks on the wheelchair even if the wheelchair is fixed in the docking station. This prevents uncontrolled rolling of the wheelchair at the end of the journey after the docking station is unlocked.

Putting on the occupant restraint system

- 1) Put on the 3-point occupant restraint system installed in the vehicle.
- 2) Ensure that
 - the lap belt runs correctly, fully and as low as possible over the front of the pelvis (see fig. 9, item 1)
 - the shoulder strap is positioned correctly and centred over the shoulder (see fig. 9, item 2)
 - the anchor points/belt buckles are positioned between 10 mm and 60 mm outside the wheels (see fig. 9)
 - the occupant restraint belts are at an angle of between 30° and 75° to the vehicle floor (see fig. 11); preferably, an angle of between 45° and 75°

Please note: An angle greater than 75° is not permitted!

- 3) Position the belts of the occupant restraint system close to the user's body (see fig. 9, details circled). The belts must not be routed over the side panels, wheels, etc. (see fig. 10, details circled).
- 4) Check that the belt strap is not twisted but rests flat against the user's body.
- 5) When the 3-point occupant restraint system is applied, check that the buckle is positioned such that the release button cannot be opened unintentionally through contact with vehicle parts or wheelchair parts while driving or in the event of an accident.
- 6) Tighten the 3-point occupant restraint system installed in the vehicle as firmly as possible after it is applied, unless this is done automatically by means of a retractor.

9 Cleaning

Cleaning the edge protection

- Use a moist cloth for cleaning.

Cleaning the parts of the Dahl docking system

When cleaning the components of the Dahl Docking System, observe the information in the instructions for use for the docking system.

10 Maintenance

When maintaining the components of the Dahl Docking System, observe the information and specifications in the instructions for use for the docking system.

11 Repair

WARNING

Prohibited repairs

Severe user injuries, damage to the product due to adjustment and installation errors

- All repairs may only be carried out by qualified personnel.

Repair instructions for the attachments supplied by Ottobock

- In case of damage, promptly contact the qualified personnel who adapted the product.

Repair instructions for the “Dahl Docking System” option

- In case of damage, promptly contact the qualified personnel who adapted the product or the manufacturer's service department (for the contact details, see the instructions for use for the docking system).

12 Disposal

Return the product to the qualified personnel for disposal.

All components of the product must be disposed of properly in accordance with the respective national environmental regulations.

13 Legal information

13.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

13.2 Warranty

Further information on the warranty terms and conditions is available from the qualified personnel that adapted this product or the manufacturer's service department.

13.3 Local Legal Information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

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